P20

BLOOD PRESSURE MONITORING SYSTEM

Quick Start User Guide

For self-testing. Read instructions before use.
 Distributed by **Fora Care**
Fora Care Inc.
 810 Lawrence Dr, Suite 104, Newbury Park, CA 91320 USA
 Products made in Taiwan
 Toll Free:
 1-888-307-8188 (8:30am~5:00pm PST, Mon.-Fri.)
 1-866-469-2632 (24 Hours, 7 days/week)
 www.foracare.com/usa

315-3132000-001

Dear FORA P20 System Owner:


Thank you for choosing FORA P20 Blood Pressure Monitoring System to take control of your blood pressure. Being a home care device, this system is helpful in tracking your blood pressure values.

Your Quick Start User Guide assists you in setting up the system and running the test. Please read it carefully before use. For complete information on system use and features, please refer to the Owner's Manual.

Front View of Monitor

Rear View of Monitor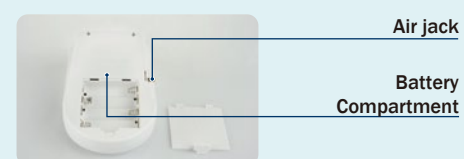

1. Set The Monitor

1.1 Press **S** to start setting. Year flashes first.

1.2 Press **▲▼** to make changes. Number can be advanced faster by holding down **▲▼**. Press **S** to proceed.

1.3

Date flashes. Follow **1.2** to make changes. The sequence of settings is:
 (1) the date
 (2) time format
 (3) time
 (4) unit of measurement
 (5) voice volume
 (6) memory deletion
 (7) language

2. Take a Single Measurement

Always apply the pressure cuff before turning on the monitor. For how to apply the pressure cuff, please see the Owner's Manual for complete information.

2.1 Press **⏻**. All the LCD symbols will appear. Press **▲▼** button to select user "1 2 3 4" (numbers from 1 to 4) whose test reading will be stored in memory. Then the cuff will begin to inflate automatically.

2.2 The heart symbol "♥" will flash when a pulse is detected during the inflation.

2.3 After the measurement, the monitor displays the systolic pressure, diastolic pressure, and pulse rate.

2.4 Press **⏻** to switch off. Or it will switch off automatically after left idle for 3 minutes.

NOTE

- If you press **⏻** during measurement, the monitor will be turned off.
- If the pulse rate symbol is shown as "♥" instead of "♥", this indicates that the monitor has detected an irregular heart beat.

3. Measurement Average Mode

Always apply the pressure cuff before turning on the monitor.

3.1 Press and hold **⏻** for three seconds. The monitor will turn on and enter the averaging mode. Press **▲▼** to select user "1 2 3 4" (numbers from 1 to 4) whose test reading will be stored in memory. Then the cuff will begin to inflate automatically.

3.2 After the first measurement is completed, the monitor will start counting down before the second measurement begins. The number on the right represents the remaining countdown between each measurement. The monitor will take three (3) measurements consecutively with an interval of 20 seconds.

3.3 After taking three measurements, the results are averaged to produce the blood pressure measurement. Press **⏻** to turn off the monitor.

4. Auscultatory Mode

Only trained healthcare professionals can perform manual operation.

4.1 Place a stethoscope on the patient's arm where there is a pulse. Wrap the cuff around the patient's upper arm and hold in place with velcro.

4.2 Press **⏻** button to turn on the monitor then press **M** to enter the Auscultatory Mode. A stethoscope symbol will appear on the display with a beep. Then the cuff begins to inflate automatically.

4. Auscultatory Mode

4.3 The operator can adjust the inflation pressure if necessary. During the measurement, press **▼** at any time to stop the inflation and start deflation, or press **▲** to inflate again. After reaching the cuff pressure, the deflation begins.

4.4 The systolic pressure is measured when the operator first hears the pulse. This reading indicates the subject's systolic pressure.

4.5 The diastolic pressure is measured from the moment the operator is unable to hear the sound of the pulse. This reading indicates the subject's diastolic pressure.

To turn off the monitor, press **⏻**.

5. Read Your Result

Reference values:

Classification	Systolic Pressure (mmHg)	Diastolic Pressure (mmHg)
Hypotension* ¹	< 90	< 60
Normal* ²	< 120	< 80
Pre-hypertension* ²	120-139	80-89
Stage 1 Hypertension* ²	140-159	90-99
Stage 2 Hypertension* ²	≥ 160	≥ 100

*¹: National Heart, Lung, and Blood Institute, Diseases and Conditions
 *²: The Seventh Report of the Joint National Committee on Prevention, Detection, Evaluation, and Treatment of High Blood Pressure. NIH Publication. 2003. No. 03-5233

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