FORA[®] P20

Blood Pressure Monitoring System

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Manual del Propietario

Dear FORA P20 System Owner:

Thanks for purchasing the **FORA P20** Blood Pressure Monitoring System. This manual provides extremely important information which helps you operate this system properly. Before using this product, please read the following contents thoroughly and carefully.

Your **FORA P20** Blood Pressure Monitoring System provides you with accurate, non-invasive measurement of your blood pressure at your upper arm based on the Oscillometric method. By means of the compact size and ease of operation of this **FORA P20** Blood Pressure Monitoring System, you can monitor your blood pressure by yourself simply at any place, any time. Furthermore, the results of measurements you get from this system can help you and your healthcare professionals to monitor and adjust your treatment plan to control your blood pressure better.

Please read it carefully.

IMPORTANT SAFETY PRECAUTIONS

READ BEFORE USE

The following basic safety precautions should always be taken.

- 1.** Close supervision is necessary when the device is used by, on, or near children, handicapped persons or invalids.
- 2.** Use this device only for the intended use described in this manual.
- 3.** Do not use accessories which are not specified by the manufacturer.
- 4.** Keep the equipment and its flexible cord away from hot surfaces.
- 5.** Do not use the equipment where aerosol sprays are being used, or where oxygen is being administered.
- 6.** Do not use the device if it is not working properly or has been damaged.
- 7.** Do not use under any circumstances on neonates or infants.
- 8.** Use of this instrument in a dry environment, especially if synthetic materials are present (synthetic clothing, carpets, etc.) may cause damaging static discharges that may cause erroneous results.
- 9.** Do not use this instrument in close proximity to sources of strong electromagnetic radiation, as these may interfere with the accurate operation.
- 10.** Proper maintenance and periodic calibration are essential to the longevity of your device. If you are concerned about your accuracy of measurement, please contact local customer service for help.

KEEP THESE INSTRUCTIONS IN A SAFE PLACE

TABLE OF CONTENTS

BEFORE YOU BEGIN	1
Warnings and Precautions	1
Intended Use	1
Principle of Measurement	2
Major Features	2
Contents of System	3
Monitor Overview	4
Display Screen	6
Setting the Monitor	7
TESTING YOUR BLOOD PRESSURE	10
Before Measurement	10
Fitting the Cuff Correctly	11
Proper Measurement Position	12
Taking Measurements	13
MONITOR MEMORY	17
Reviewing Test Results	17
DOWNLOADING THE RESULTS	18
Data Transmission via Bluetooth	18
MAINTENANCE	20
Battery	20
SYSTEM TROUBLESHOOTING	23
Error Message	23

Troubleshooting	24
DETAILED INFORMATION	25
Reference Values	25
SPECIFICATIONS	26
FEDERAL COMMUNICATIONS COMMISSION (FCC) STATEMENT	28

BEFORE YOU BEGIN

Warnings and Precautions

- The **FORA P20** Blood Pressure Monitoring System shall only be used for the testing on people over 18 years of age and over.
- This device does NOT serve as a cure for any symptoms or diseases. The data measured are for reference only. Always consult your physician to have the results interpreted.
- This device is NOT able to take measurements in the presence of common arrhythmia, such as arterial or ventricular premature beats or arterial fibrillation. It may produce reading error.
- Do NOT use the device for purposes other than measuring blood pressure for human beings.
- Do NOT apply the cuff to areas other than your arm.

Intended Use

The intended use of **FORA P20** Blood Pressure Monitoring System is to measure human systolic, diastolic blood pressure and heart rate by using the oscillometric method. The measurement position of the device is the upper arm of the subject. This system should only be used for the testing on people over 18 years of age and over.

Principle of Measurement

Blood pressure is measured non-invasively at the upper arm based on the Oscillometric method. Blood pressure measurements determined with this device are equivalent to those obtained by a trained observer using the cuff/stethoscope auscultation method, within the limits prescribed by the American National Standard, Electronic or automated sphygmomanometers.


Major Features

This system has the following distinguishing features and provides you and your doctor to manage your health effectively.

1. Auscultatory Mode - provide an adjunct function for trained persons to measure blood pressure by hearing.
2. Multiuser function - provide users to store their results in the memory individually.

Contents of System

The **FORA P20** Blood Pressure Monitoring System kit includes: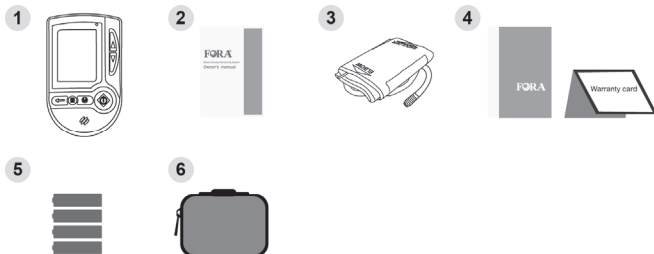


- ① 1 x Monitor
- ② 1 x Owner's Manual
- ③ 1 x Pressure Cuff
- ④ 1 x Quick Start User Guide / Warranty Card
- ⑤ 4 x AA Alkaline Batteries
- ⑥ 1 x Protective Wallet

NOTE:

- If any items are missing from your kit or opened before use, please contact local customer service or place of purchase for assistance.

Monitor Overview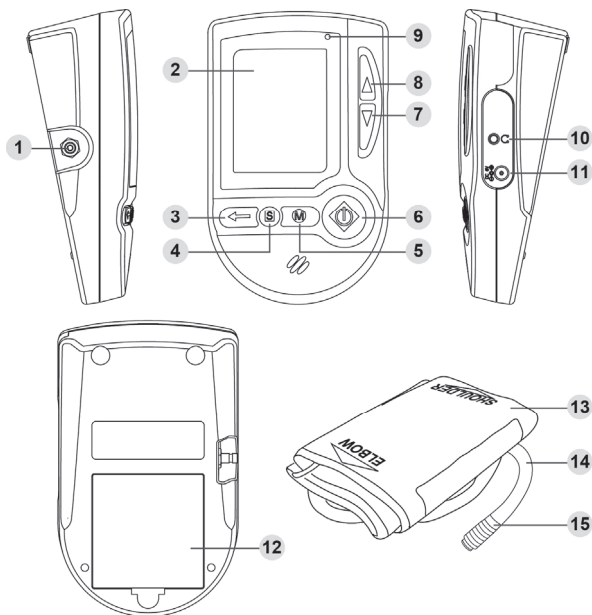


- | | |
|---|------------------------------|
| ① Air Jack | ⑧ Up Button |
| ② Display Screen | ⑨ Bluetooth Indicator |
| ③ Repeat/Pairing Button
Press to repeat the voice instructions/pair with personal device. | ⑩ Earphone Jack |
| ④ Set Button (S) | ⑪ AC/DC Adapter Port |
| ⑤ Main Button (M) | ⑫ Battery Compartment |
| ⑥ On/Off Button | ⑬ Pressure Cuff |
| ⑦ Down Button | ⑭ Air Tube |
| | ⑮ Air Plug |

NOTE:

- If you prefer to listen to the voice messages by earphone, plug a pair of earphones into the earphone jack on the side of your monitor.
- This monitor has some speaking functions but is not intended for use by the visually impaired.

Display Screen

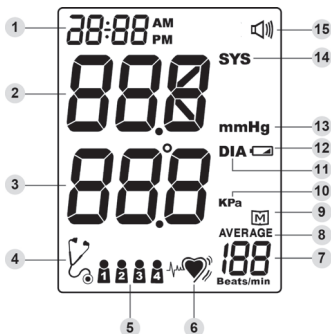- | | |
|-----------------------------------|-------------------------------------|
| ① Date and Time | ⑨ Memory Mode Symbol |
| ② Systolic Pressure Value | ⑩ Unit of Measurement (KPa) |
| ③ Diastolic Pressure Value | ⑪ Diastolic Pressure Symbol |
| ④ Auscultatory Mode Symbol | ⑫ Battery Symbol |
| ⑤ User Number | ⑬ Unit of Measurement (mmHg) |
| ⑥ Pulse Rate Symbol | ⑭ Systolic Pressure Symbol |
| ⑦ Pulse Rate | ⑮ Volume Symbol |
| ⑧ Three Time Average | |

Setting the Monitor

Before using your monitor for the first time, check and update these settings. Make sure you complete the steps below and save the desired settings.

Entering the Setting Mode

Start with the monitor off. Press **S** until the monitor turns on.

1. Setting the date

With the year flashing, press **▲▼** to select the correct year. Press **S**.

With the month flashing, press **▲▼** to select the correct month. Press **S**.

With the day flashing, press **▲▼** to select the correct day. Press **S**.

2. Setting the time format


Press **▲▼** to select the desired time format. Press **S**.

3. Setting the time

With the hour flashing, press **▲▼** to select the correct hour. Press **S**.

With the minute flashing, press **▲▼** to select the correct minute. Press **S**.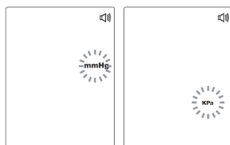

4. Setting the unit of measurement

Press **▲▼** to select **mmHg** or **Kpa**.

Press **S**.

5. Setting the speaking volume

There are seven (7) speaking volume options you can choose from. Press **▲▼** to select the desired speaking volume. Press **S**.

Volume 0 indicates that the speaking function is turned off, and will not display “”. Volume 1 to 7 indicates speaking volume from low to high, and “” will be displayed during testing.

6. Deleting the memory

While “dEL” and a blinking “” symbol appear on the display, if you do not wish to delete the saved results, press **S** to skip this step.

If you wish to delete ALL the results, press **M** twice. “**CL_{ALL}**” and “**[M]**” are displayed on the monitor, which indicates that all results have been deleted.

7. Choosing a language

Press **▲▼** to select L1/L2. The default language for the monitor is L1 (English). Press **S**.

Then the monitor will turn off automatically or you can press **⏻** to turn off the monitor.

Congratulations! You have completed all settings.

NOTE:

- These settings can **ONLY** be changed in the setting mode.
- If the monitor is idle for 3 minutes during the setting mode, it will switch off automatically.

TESTING YOUR BLOOD PRESSURE


This monitor provides you three different ways to measure your blood pressure. Select from the operation below:

- **Single measurement**
Perform an individual blood pressure measurement.
- **Averaging measurement**
Automatically perform three (3) consecutive blood pressure measurements and display the average result at the end.
- **Auscultatory Mode measurement**
Only trained health-care professionals may use this mode to measure blood pressure manually with a stethoscope by listening to Korotkoff sounds.


Before Measurement

- Avoid caffeine, tea, alcohol and tobacco for at least 30 minutes before measurement.
- Wait for 30 minutes after exercising or bathing before measurement.
- Sit or lie down for at least 10 minutes before measurement.
- Do not measure when feeling anxious or tense.
- Take a 5-10 minute break between measurements. This break can be longer if necessary, depending on your physical conditions.
- Keep the records for your doctor as reference.
- Blood pressure varies between each arm. Always measure your blood pressure on the same arm.


Fitting the Cuff Correctly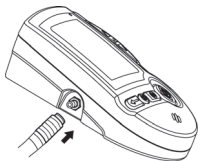

1. Connect the air plug of the tubing to the air jack of the monitor.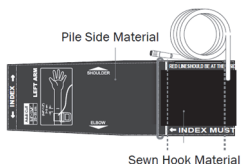


2. Assemble the cuff as shown on the right. The smooth surface should be inside the cuff loop and the metal D-ring should not touch your skin.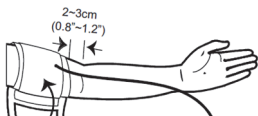

3. Stretch your left (right) arm in front of you with your palm facing up. Slide and place the cuff onto your arm with the air tube and artery mark region (in red) toward the lower arm.

Wrap and tighten the cuff above your elbow. The red line on the edge of the cuff should be approximately 0.8 to 1.2 inches (2 to 3 cm) above your elbow. Align the tube over the main arteries on the inside.

4. Leave a little free space between the arm and the cuff, you should be able to fit 2 fingers between them. Clothing must not restrict the arm. Remove all clothing covering or constricting the measurement arm.

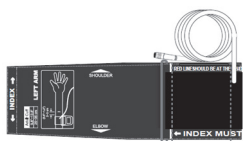► The range index of cuff should fall into this range.

5. Press the hook material firmly against the pile material. The top and bottom edges of the cuff should be tightened evenly around your upper arm.

Proper Measurement Position

1. Sit down for at least 10 minutes before measuring.

2. Place your elbow on a flat surface. Relax your hand with the palm facing up.

3. **Make sure the cuff is about the same height as the location of your heart.** Press . Remain still and do not talk or move during the measurement.

WARNING:

- If the cuff is relatively lower/higher than the heart, the obtained blood pressure value could be higher/lower than the actual value. A 15 cm difference in height may result in an error around 10 mmHg.

4. Measurement is in progress.


After the monitor is turned on, the cuff will begin to inflate automatically.

Taking Measurements

This monitor provides you three different ways to measure your blood pressure.

• Taking a Single Measurement

Always apply the pressure cuff before turning on the monitor.

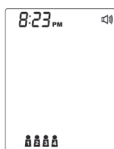1. Press . All the LCD symbols will appear. Press to select user "1 2 3 4" (numbers from 1 to 4) whose test reading will be stored in memory. Then the cuff will begin to inflate automatically.

**Voice: Thank you for using FORA Telehealth product, please relax during measurement.*

2. The heart symbol "♥" will flash when a pulse is detected during the inflation.

3. After the measurement, the monitor displays the systolic pressure, diastolic pressure and pulse rate.

**Voice: Your systolic blood pressure is (number) millimeters of mercury. Your diastolic blood pressure is (number)*

millimeters of mercury. Your heart rate is (number) beats per minute.


4. Press  to switch off. Or it will switch off automatically after left idle for 3 minutes.

NOTE:

- If you press  during measurement, the monitor will be turned off.
- If the pulse rate symbol is shown as “” instead of “”, this indicates that the monitor has detected an irregular heart beat.

• **Averaging Measurement Mode**

Always apply the pressure cuff before turning on the monitor.

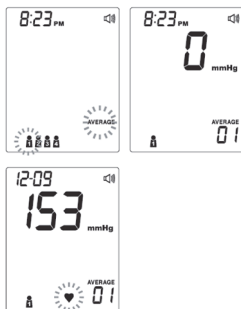1. Press and hold  for three seconds. The monitor will turn on and enter the averaging mode. Press   to select user “   ” (numbers from 1 to 4) whose test reading will be stored in memory and press  to confirm. Then the cuff will begin to inflate automatically.

**Voice: Thank you for using FORA Telehealth product, please relax during measurement.*

2. After the first measurement is completed, the monitor will start counting down before the second measurement begins. The number on the right represents the remaining countdown between each measurement. The monitor will take three (3) measurements consecutively with intervals of 20 seconds.

3. After taking three measurements, the results are averaged to produce the blood pressure measurement. Press  to turn off the monitor.

**Voice: Your systolic blood pressure is (number) millimeters of mercury. Your diastolic blood pressure is (number) millimeters of mercury. Your heart rate is (number) beats per minute.*

• Auscultatory Mode


Only trained healthcare professionals may perform this manual blood pressure measurement.

This manual method involves applying a stethoscope to patient's arm and listening to the pulse while the air is slowly let out again.

The systolic pressure is the maximum pressure in an artery at the

moment when the heart is beating and pumping blood through the body. The diastolic pressure is the lowest pressure in an artery in the moments between beats when the heart is resting.

- 1.** Place a stethoscope on the patient's arm where there is a pulse. Apply the cuff to the patient's upper arm and hold in place with Velcro.

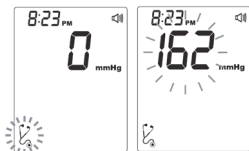- 2.** Press  and wait for the user number to appear. Press  to switch to the auscultatory mode. A auscultatory mode symbol will appear on the display with a beep. The cuff begins to inflate automatically.

- 3.** After reaching the cuff pressure, the deflation begins. The operator can adjust the inflation pressure if necessary. During the measurement, press and hold  at any time to keep inflating, release at the desired pressure.

- 4.** The systolic pressure is measured when the operator first hears the pulse. Take the reading according to the value on the display. This reading indicates the patient's systolic pressure.

- 5.** The diastolic pressure is measured from the moment the operator is unable to hear the sound of the pulse. Take the reading according to the value on the display. This reading indicates the patient's diastolic pressure.

MONITOR MEMORY

Your monitor stores the 400 most recent test results along with respective date and time in its memory. To enter the monitor memory, **start with the monitor switched off.**

Reviewing Test Results

1. Press and release **M**

“**M**” will appear on the display. The user symbol (numbers from 1 to 4) appears first, press **▲▼** to select which one you used to store in the memory.

2. Press **▲▼**

Review all test results stored in the monitor.

3. Exit the memory

Press **⏻** and the monitor will turn off.

NOTE:

- Any time you wish to exit the memory, press **⏻** or leave it idle for 3 minutes. The monitor will turn off automatically.
- If using the monitor for the first time, “—” displays when you recall the test results or review the average result. This indicates that there is no test result in the memory.

DOWNLOADING THE RESULTS

Data Transmission via Bluetooth

You can transmit your measurement data from the monitor to an App on your mobile device via Bluetooth signal. Please note that you must first pair your monitor to your mobile device before attempting to transmit any data. Please contact customer service or the place of purchase for further assistance.

Pairing with your mobile device

1. Press and hold  on the monitor until the Bluetooth indicator starts to flash.
2. On your mobile device, turn on the Bluetooth function and follow the instruction on the App to pair the device (e.g., Search for monitor to add it into App).
3. After successfully pairing the App to your monitor, the Bluetooth function on the monitor will then start to transmit the data to your App.

Bluetooth indicator on the monitor

BLUETOOTH INDICATOR	STATUS
Flashing Blue	The Bluetooth function is enabled and waiting for connection.
Solid Blue	The Bluetooth connection is established.

NOTE:

- While the monitor is in transmission mode, it will be unable to perform a test.
- Make sure that your Bluetooth enabled device has turned on the Bluetooth signal prior to transmitting the data, and that the monitor is within range.
- For OS version requirements please refer to the App description on the Apple

App Store or Google Play Store.

- A compatibility issue between your mobile device and the monitor might occur due to the way Bluetooth functionality is implemented by the various mobile device manufacturers.

MAINTENANCE

Battery

Your monitor comes with four 1.5V AA size alkaline batteries.

Low Battery Signal

The monitor will display the message below to alert you when the monitor power is getting low.

The “” appears along with display messages:

The monitor is functional and the result remains accurate, but it is time to change the batteries.

The “” appears with **E-b** and low:

The power is not enough to do a test. Please change the batteries immediately.

Replacing the Battery

To replace the batteries, make sure that the monitor is turned off.


1. Press the edge of the battery cover and lift it up to remove.
2. Remove the old batteries and replace it with four 1.5V AA size alkaline batteries.
3. Close the battery cover.


NOTE:

- Do not use different type, date or brand name batteries together. Use only new batteries of the required size and type.
- Replacing the batteries does not affect the test results stored in the memory.
- As with all small batteries, these batteries should be kept away from children. If swallowed, promptly seek medical assistance.
- Batteries might leak chemicals if unused for a long time. Remove the batteries if you are not going to use the device for an extended period (i.e., 3 months or more).
- Properly dispose of the batteries according to your local environmental regulations.

Connect AC adapter to the monitor

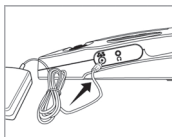
- 1.** Connect AC adapter plug to AC adapter jack of the monitor.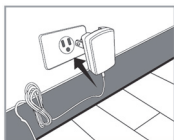


- 2.** Plug AC adapter power plug into an electrical outlet. The monitor is ready for use.

Remove AC adapter from the monitor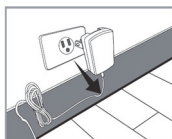

- 1.** When the monitor is off, remove AC adapter plug from the electrical outlet.

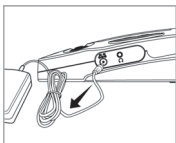2. Disconnect AC adapter plug from AC adapter jack on the monitor.

To prevent the monitor from dirt, dust or other contaminants, please wash and dry your hands thoroughly before use.

Cleaning

- To clean the monitor exterior, wipe it with a cloth moistened with tap water or a mild cleaning agent, then dry the device with a soft dry cloth. Do **NOT** flush it with water.
- Do **NOT** use organic solvents to clean the monitor.
- Do **NOT** wash the pressure cuff.
- Do **NOT** iron the pressure cuff.

Monitor Storage

- Storage condition: -4 °F to 140 °F (-20 °C to 60 °C), below 95% relative humidity.
- Always store or transport the monitor in its original storage case.
- Avoid dropping and heavy impact.
- Avoid direct sunlight and high humidity.

Monitor Disposal

The batteries in this used monitor should be removed and the monitor should be disposed in accordance with local regulations.

SYSTEM TROUBLESHOOTING

If you follow the recommended action but the problem persists, or error messages other than the ones below appear, please call your local customer service. Do not attempt to repair yourself and never try to disassemble the monitor under any circumstances.

Error Message

MESSAGE	WHAT IT MEANS	WHAT TO DO
E-1	Inflation or pressure error.	Please contact local customer service for help.
E-2	Deflation error.	
E-4	Blood pressure measurement error.	Refit cuff tightly and correctly. Relax and repeat measurement. If error still exists, please contact the local customer service for help.
E-b	Battery is too low.	Replace the batteries or connect 6V AC/DC adapter.
E-E E-A E-3 E-C	Problem with the monitor.	Review the instructions and repeat the test. If the monitor still does not work, please contact the local customer service for help.

Troubleshooting

1. If nothing is display after pressing :

POSSIBLE CAUSE	WHAT TO DO
Battery exhausted.	Replace the batteries.
Batteries incorrectly installed or absent.	Check that the batteries are correctly installed.

2. If the heart rate is higher/lower than user's average:

POSSIBLE CAUSE	WHAT TO DO
Movement during measurement.	Repeat measurement.
Measurement taken just after exercise.	Rest at least 30 minutes before repeating measurement.

3. If the result is higher/lower than user's average measurement:

POSSIBLE CAUSE	WHAT TO DO
May not be in correct position while measuring.	Adjust to the correct position to measure.
Blood pressure naturally varies from time to time.	Keep in mind for next measurement.

4. If the cuff inflates again while measuring:

POSSIBLE CAUSE	WHAT TO DO
Cuff is not fastened.	Fasten the cuff again.
If user's blood pressure is higher than the pressure the device has inflated, the device will automatically increase the pressure and start to inflate again. Stay relaxed and wait for the measurement.	

DETAILED INFORMATION

Reference Values

Clinical studies show that adult diabetes is often accompanied by elevated blood pressure. People with diabetes can reduce their heart risk by managing their blood pressure along with diabetes treatment*1.

Knowing your routine blood pressure trend tells you whether your body is in good condition or not. Human blood pressure naturally increases after reaching middle age. This symptom is a result of continuous aging of the blood vessels. Further causes include obesity, lack of exercise and cholesterol (LDL) adhering to the blood vessels. Rising blood pressure accelerates hardening of the arteries, and the body becomes more susceptible to apoplexy and coronary infarction. The recommended blood pressure range is presented below:

Classification	Systolic Pressure (mmHg)	Diastolic Pressure (mmHg)
Hypertension*2	< 90	< 60
Normal*3	< 120	< 80
Elevated*3	120 - 129	< 80
Stage 1 Hypertension*3	130 - 139	80 - 89
Stage 2 Hypertension*3	≥ 140	≥ 90
Hypertensive Crisis*3	≥ 180	≥ 120

Reference

*1: National Heart, Lung, and Blood Institute, Diseases and Conditions.

*2: The Seventh Report of the Joint National Committee on Prevention, Detection, Evaluation, and Treatment of High Blood Pressure. NIH Publication. 2003. No. 03-5233

*3: New ACC (American College of Cardiology) and AHA (American Heart Association) guidelines for the detection, prevention, management and treatment of high blood pressure (2017).

SPECIFICATIONS

Model No.: FORA P20

Dimension: 152 (L) x 99 (W) x 55 (H) mm

Weight: 280 g (without battery)

Power Source: four 1.5V AA alkaline batteries

Display: LCD

Memory: 400 measurement results (100 per user)

External Output: Bluetooth

Power Saving: Automatic power off if system is idle for 3 minutes (normal mode) or 5 minutes (RF mode)

System Operating Condition:

50°F to 104°F (10°C to 40°C), below 85% R.H.

Monitor Storage/Transportation Conditions:

-4°F to 140°F (-20°C to 60°C), below 95% R.H.

Power Supply Input: DC +6V / 1A (max) via Power Plug

Expected Service Life: 3 years

● Blood Pressure Measurement Performance

Pressure Range: 0 – 300 mmHg

Measurement Unit: mmHg or KPa

Systolic Measurement Range: 60 – 255 mmHg

Diastolic Measurement Range: 30 – 195 mmHg

Pulse Rate Measurement Range: 40 – 199 beats / minute

Maximum Inflation Pressure: 300 mmHg

Accuracy of Pressure: ± 3 mmHg or $\pm 2\%$ of reading

Accuracy of Pulse Rate: $\pm 4\%$ of reading

This device has been tested to meet the electrical and safety requirements of: IEC/EN 60601-1, IEC/EN 60101-1-2, EN 300 328.

Reference to Standards:

- EN 1060-1 /-3, NIBP-requirements
- IEC60601-1 General requirement for safety
- IEC60601-1-2 Requirements for EMC
- EN1060-4, NIBP clinical investigation
- AAMI / ANSI / IEC 80601-2-30, ANSI/AAMI/ISO 81060-2, NIBP requirements
- FCC part 15C

FEDERAL COMMUNICATIONS COMMISSION (FCC) STATEMENT

15.21

You are cautioned that changes or modifications not expressly approved by the part responsible for compliance could void the user's authority to operate the equipment.

15.105(b)

Federal Communications Commission (FCC) Statement

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

- 1) This device may not cause harmful interference and
- 2) This device must accept any interference received, including interference that may cause undesired operation of the device.

FCC RF Radiation Exposure Statement:

- 1) This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.
- 2) This equipment complies with FCC RF radiation exposure limits set forth for an uncontrolled environment. This equipment should be installed and operated with a minimum distance of 20 centimeters between the radiator and your body.

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Distributed by ForaCare, Inc. / Distribuido por ForaCare, Inc.