FORA[®] P50

BLOOD PRESSURE MONITORING SYSTEM

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Owner's Manual
Manual del Propietario

Dear **FORA[®] P50** System Owner:

Thank you for purchasing the **FORA P50** Blood Pressure Monitoring System. This manual provides important information which helps you to operate this system properly. Before using this product, please read the following contents thoroughly and carefully.

According to the clinical studies from American Diabetes Association, elevated blood pressure often accompanies adult diabetes patients. These studies also suggest that diabetes patients could reduce the risk of cardiovascular diseases by managing their blood pressure. With the use of this **FORA P50** Blood Pressure Monitoring System, you can easily monitor your blood pressure by yourself. In addition, this system can help you and your healthcare professionals to monitor and adjust your treatment plans, and keep your blood pressure under control.

If you have other questions regarding this product, please contact the place of purchase.

IMPORTANT SAFETY INSTRUCTIONS

READ BEFORE USE

1. Use this device **ONLY** for the intended use described in this manual.
2. Do **NOT** use accessories which are not specified by the manufacturer.
3. Do **NOT** use the device if it is not working properly or damaged.
4. Do **NOT** use under any circumstances on neonates or infants.
5. This device does **NOT** serve as a cure for any symptoms or diseases. The data measured are for reference only. Always consult your doctor to have the results interpreted.
6. Keep the equipment and its flexible cord away from hot surfaces.
7. Do **NOT** apply the cuff to areas other than the place directed.
8. Use of this instrument in a dry environment, especially if synthetic materials are present (synthetic clothing, carpets etc.) may cause damaging static discharges that may cause erroneous results.
9. Do not use this instrument in close proximity to sources of strong electromagnetic radiation, as these may interfere with the accurate operation.
10. Proper maintenance and periodic calibration are essential to the longevity of your device. If you are concerned about your accuracy of measurement, please contact local customer service for help of re-calibration after two years of service.

KEEP THESE INSTRUCTIONS IN A SAFE PLACE

TABLE OF CONTENTS

BEFORE YOU BEGIN	06
Intended Use	06
Test Principle	06
Contents of System	07
Monitor Overview	08
Screen Display	09
Speaking Function	10
SETTING THE MONITOR	11
User Number Selection	15
TESTING YOUR BLOOD PRESSURE	16
Before Measurement	16
Taking Measurements	19
MONITOR MEMORY	22
Reviewing Test Results	22
MAINTENANCE	24
Battery	24
Using AC Adapter	26
Caring for Your Monitor	27
SYSTEM TROUBLESHOOTING	28
Result Indicator	28
Error Messages	30
Troubleshooting	31
DETAILED INFORMATION	32
Reference Values	32
SPECIFICATIONS	33

BEFORE YOU BEGIN

Intended Use

The **FORA P50** Blood Pressure Monitoring System is intended to be used to measure the systolic and diastolic blood pressure and pulse rate by using a non-invasive technique in which an inflatable cuff is wrapped on the upper arm. This system should only be used for the testing of people who are 18 years old or older. The device is not to be used for the diagnosis or screening of hypertension or for testing on neonates. It is intended for use at home and in clinical settings.

This monitor has some speaking function but is not intended for use by the visually impaired.

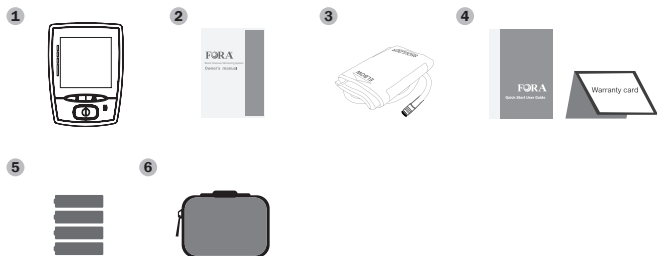
Test Principle

Blood pressure is measured non-invasively at the arm based on oscillometric method.

This device is NOT able to take measurements in the presence of common arrhythmia, such as arterial or ventricular premature beats or arterial fibrillation. It may produce reading error.

Contents of System

Your new **FORA P50** system kit includes:


- | | |
|---|---------------------------|
| 1 Monitor | 5 Batteries |
| 2 Owner's Manual | 6 Protective Storage Case |
| 3 Pressure Cuff (arm type) | |
| 4 Quick Start User Guide /
Warranty Card | |

NOTE


If any items are missing from your kit or opened before use, please contact local customer service or place of purchase for assistance.

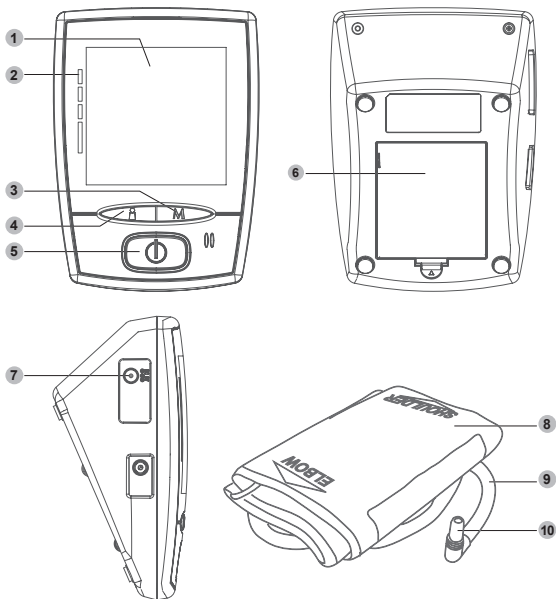Monitor Overview

- 1 Display Screen**
- 2 Classification Indicator**
Please see Result Indicator for detailed information.
- 3 M Button**
Enter the meter memory.
- 4 User Selection Button**
Assist you with user number selection.
- 5 On/Off Button**
- 6 Battery Compartment**
- 7 DC Adapter Port**
Connect to a power supply.
- 8 Pressure Cuff**
- 9 Air Tube**
- 10 Air Plug**
Connect to Air Jack.

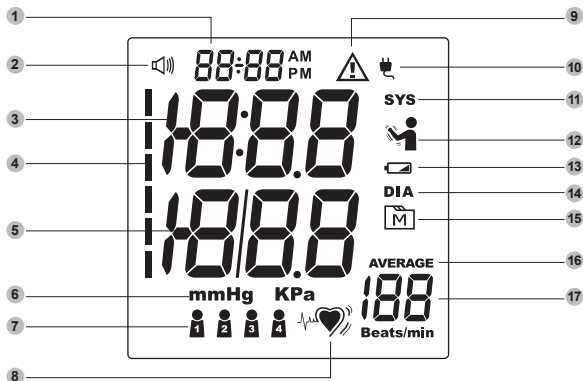Screen Display

- | | |
|----------------------------|------------------------------|
| 1 Date/ Time | 10 Adapter symbol |
| 2 Voice volume symbol | 11 Systolic pressure symbol |
| 3 Systolic pressure value | 12 Special message |
| 4 Indication bars | 13 Low battery symbol |
| 5 Diastolic pressure value | 14 Diastolic pressure symbol |
| 6 Units of blood pressure | 15 Memory mode symbol |
| 7 User number | 16 Three time average |
| 8 IHB detection symbol | 17 Pulse rate |
| 9 Error message | |

Speaking Function

FORA P50 “speaks” aloud with step-by-step instructions to guide you through the process of using the meter. The following table tells you when and what the monitor “speaks”.

WHEN does the monitor say?	WHAT does the monitor say?
Using the monitor	
When turning on the monitor	Thank you for using this product. Please relax during measurement.
When power is insufficient.	Battery is dead, please replace.
When the test is completed or reviewing the memory. (result appears on display)	The systolic blood pressure is (number) millimeter of mercury. The diastolic blood pressure is (number) millimeter of mercury. The heart rate is (number) beats per minute.

SETTING THE MONITOR

Before using your monitor for the first time, check and update these settings. Make sure you complete the steps below and save the desired settings.

► Entering the setting mode

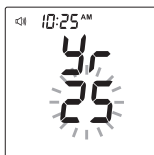
Start with the monitor off. Press and firmly hold **i** until the monitor turns on.


Step 1

Setting the date.


With the year flashing, press **M** to select the correct year. Press **i** to set.

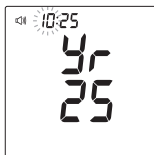With the month flashing, press **M** to select the correct month. Press **i** to set.

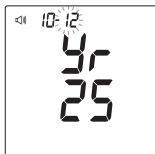
With the day flashing, press **M** to select the correct day. Press **i** to set.


Step 2

Set the time.

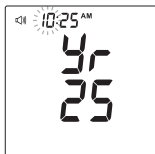With the hour flashing, press **M** to select the correct hour. Press **i** to set.

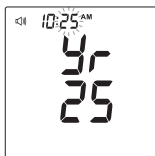
With the minute flashing, press **M** to select the correct minute appears. Press **i** to set.

Step 3

Set the time format.


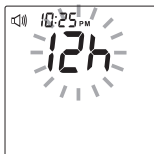
Press **M** to select the desired time format — 12h or 24h. Press **i** to set.

Step 4

Set the unit of measurement.

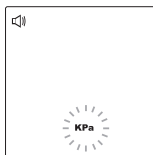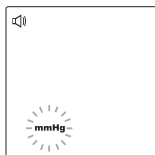Press **M** to select **mmHg** or **Kpa**. Press **i** to set.


Step 5

Setting the speaking volume.

There are seven (7) speaking volume options you can choose from.

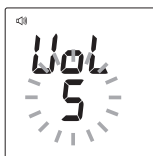Press **M** to select the desired speaking volume. Press **i** to set.



Volume 0 indicates that the speaking function is turned off, and will not display “”. Volume 1 to 7 indicates speaking volume from low to high, and “” will be displayed during testing.

Step 6

Choosing a language.

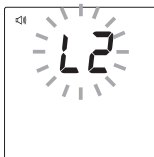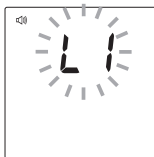Press **M** to select L1/L2. The default language for the monitor is L1, which is English. Press **i** to set.


Step 7

Delete the memory.

While "dEL" and a blinking **M** symbol appear on the display, if you do not wish to delete the saved results, press **i** to skip this step.

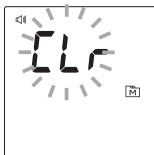If you wish to delete ALL the results, press **M** once. "CL" and **M** are displayed on the monitor, which indicates that all results have been deleted.

Congratulations! You have completed all settings!

NOTE

- These parameters can **ONLY be changed** in the setting mode.
- If the monitor is idle for 3 minutes during the setting mode, it will turn off automatically.

User Number Selection

This system stores blood pressure measurement for up to four users. Each user's test results are stored separately under each user number.

Step 1

Press  to select the desired user number.

Step 2

Press  to confirm. The monitor will display the selected number and then turn off automatically.

TESTING YOUR BLOOD PRESSURE

This monitor provides you two different ways to measure your blood pressure. Select from the operations below:

- **Single measurement**

Perform an individual blood pressure measurement.

- **Averaging measurement**

Automatically perform three (3) consecutive blood pressure measurements and display the average result at the end.

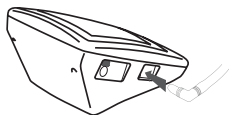
Before Measurement

- Avoid caffeine, tea, alcohol and tobacco for at least 30 minutes before measurement.
- Wait for 30 minutes after exercising or bathing before measurement.
- Sit or lie down for at least 10 minutes before measurement.
- Do not measure when feeling anxious or tense.
- Take a 5-10 minute break between measurements. This break can be longer if necessary, depending on your physical conditions.
- Keep the records for your doctor as reference.
- Blood pressure varies between each arm. Always measure your blood pressure on the same arm.

► Fitting the Cuff Correctly


Step 1

Connect the air plug of the tubing to the air jack of the monitor.


Step 2

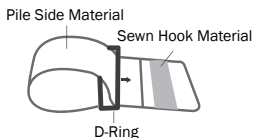Assemble the cuff as shown on the right. The smooth surface should be inside the cuff loop and the metal D-ring should not touch your skin.

Step 3

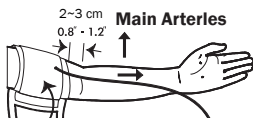Stretch your left (right) arm in front of you with your palm facing up. Slide and place the cuff onto your arm with the air tube and artery mark region (in red) toward the lower arm.


Wrap and tighten the cuff above your elbow. The red line on the edge of the cuff should be approximately 0.8 to 1.2 inches (2 to 3 cm) above your elbow. Align the tube over the main arteries on the inside.

Step 4

Leave a little free space between the arm and the cuff, you should be able to fit 2 fingers between them. Clothing must not restrict the arm. Remove all clothing covering or constricting the measurement arm.

Step 5

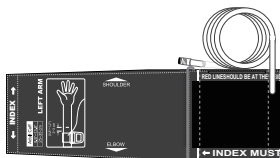Press the hook material firmly against the pile material. The top and bottom edges of the cuff should be tightened evenly around your upper arm.

► The range index of cuff should fall into this range.

► Proper Measurement Position

Step 1

Sit down for at least 10 minutes before measuring.

Step 2

Place your elbow on a flat surface. Relax your hand with the palm facing up.

Step 3

Make sure the cuff is about the same height as the location of your heart.

Press . Remain still and do not talk or move during the measurement.

Step 4

Measurement is in progress.

After the monitor is turned on, the cuff will begin to inflate automatically.

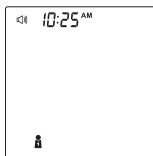
Taking Measurements

► Taking a Single Measurement

Always apply the pressure cuff before turning on the monitor.

Step 1

Press . All the LCD symbols will appear. Press  to select user “   ” (numbers from 1 to 4) whose test reading will be stored in memory. Then the cuff will begin to inflate automatically.

Step 2

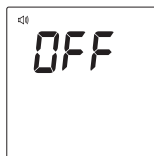
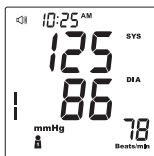
The heart symbol “” will flash when a pulse is detected during the inflation.

Step 3

After the measurement, the monitor displays the systolic pressure, diastolic pressure and pulse rate.

Step 4

Press  to switch off. Or it will switch off automatically after left idle for 3 minutes.

NOTE

- If you press  during measurement, the monitor will be turned off.
- If the pulse rate symbol is shown as “” instead of “”, this indicates that the monitor has detected an irregular heart beat.
- If the screen shows as below instead of blood pressure values, this indicates that the meter has detected a big movement or great vibration(s) of the arm during the measurement and cannot produce a value. The meter will then re-pump automatically and take another measurement. Please remain still while taking the measurement.

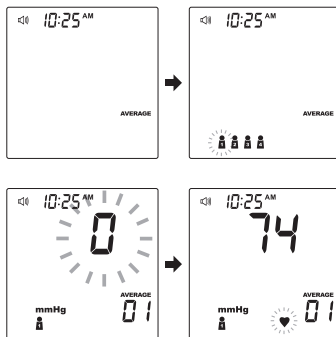
► Averaging Measurement Mode

Always apply the pressure cuff before turning on the monitor.


Step 1

Press and hold  for seconds. The monitor will turn on and enter the **AVERAGE** mode. Press  to select user “” (numbers from 1 to 4) whose test reading will be stored in memory and press  to confirm. Then the cuff will begin to inflate automatically.


Step 2

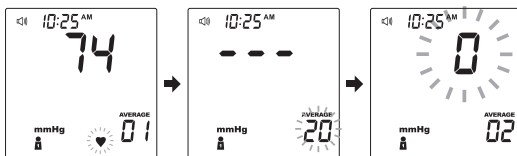After the first measurement is completed, the monitor will start counting down before the second measurement begins. The number on the right represents the remaining countdown between each measurement. The monitor will take three (3) measurements consecutively with intervals of 20 seconds.

Step 3

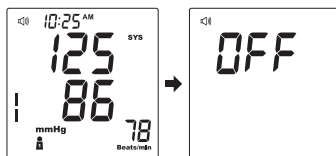After taking three measurements, the results are averaged to produce the blood pressure measurement. Press  to turn off the monitor.

MONITOR MEMORY

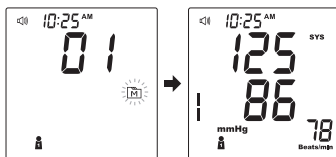
Your monitor stores the 120 most recent test results along with respective dates and times in its memory. To recall the memory, **start with the monitor off.**

Reviewing Test Results

Step 1

Press and release **M**.

"**M**" will appear on the display. The user symbol (numbers from 1 to 4) appears first, press **i** to select which one you used to store in the memory.

Step 2

Press **M**.

Review all test results stored in the monitor.

Step 3

Exit the memory.

Press **⏻** and the monitor will turn off.

NOTE

- Any time you wish to exit the memory, press  or leave it idle for 3 minutes. The monitor will turn off automatically.
- If using the monitor for the first time, “—” displays when you recall the test results or review the average result. This indicates that there is no test result in the memory.
- Any time you wish to review the results of different user numbers, press .

MAINTENANCE

Battery

Your monitor comes with four (4) 1.5V AA size alkaline batteries.

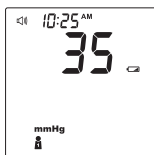
► Low Battery Signal

The monitor will display either of the two messages below to alert you when the monitor power is getting low.

Step 1

The “” symbol appears along with display messages:


The monitor is functional and the result remains accurate, but it is time to change the batteries.

Step 2

The “” symbol appears with E-b and low:

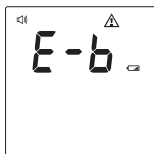There is not enough power to do a test. You must change the batteries immediately.

► Replacing the Battery

To replace the batteries, make sure that the monitor is turned off.

Step 1

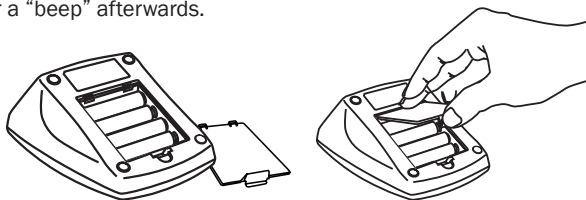
Press the edge of the battery cover and lift it up to remove.

Step 2

Remove the old batteries and replace with four 1.5V AA size alkaline batteries.

Step 3

Close the battery cover. If the batteries are inserted correctly, you will hear a “beep” afterwards.

NOTE

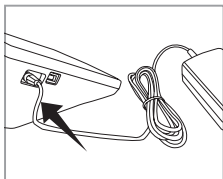
- Replacing the batteries does not affect the test results stored in memory.
- As with all small batteries, these batteries should be kept away from small children. If swallowed, promptly seek medical assistance.
- Batteries might leak chemicals if unused for a long time. Remove the batteries if you are not going to use the device for an extended period (i.e., 3 months or more).
- Properly dispose of the batteries according to your local environmental regulations.

Using AC Adapter

► Connect AC adapter to the monitor


Step 1

Connect AC adapter plug to AC adapter jack of the monitor.

Step 2

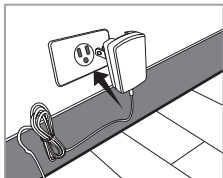
Plug AC adapter power plug into an electrical outlet. The monitor is ready for use.

► Remove AC adapter from the monitor


Step 1

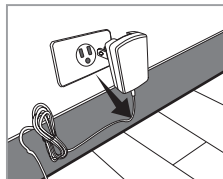When the monitor is off, remove AC adapter power plug from the electrical outlet.

Step 2

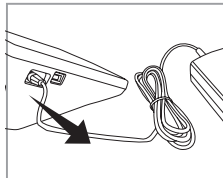Disconnect AC adapter plug from AC adapter jack on the monitor.

Caring for Your Monitor

To prevent the meter from dirt, dust or other contaminants, please wash and dry your hands thoroughly before use.

► **Cleaning**

- To clean the monitor exterior, wipe it with a cloth moistened with tap water or a mild cleaning agent, then dry the device with a soft dry cloth. Do **NOT** flush it with water.
- Do **NOT** use organic solvents to clean the monitor.
- Do **NOT** wash the pressure cuff.
- Do **NOT** iron the pressure cuff.

► **Monitor Storage**

- Storage condition: -13°F to 158°F (-25°C to 70°C), below 95% relative humidity.
- Always store or transport the monitor in its original storage case.
- Avoid dropping and heavy impact.
- Avoid direct sunlight and high humidity.

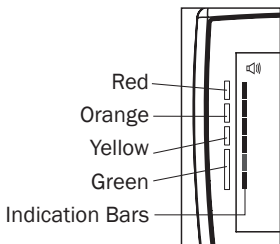
► **Monitor Disposal**

The batteries in this used meter should be removed and the meter should be disposed in accordance with local regulations.

SYSTEM TROUBLESHOOTING

If you follow the recommended actions but the problem persists, or error messages other than the ones below appear, please contact local customer service. Do not attempt to repair by yourself and never try to disassemble the monitor under any circumstances.

Result Indicator

Your test result displays one to six (1 to 6) indication bars which falls within a region marked with a specific color. There are total four (4) region-markings with different colors on the left-hand side of the meter beside the screen, green, yellow, orange and red. The region-markings are based upon the classification of 2007 ESH-ESC Practice Guidelines for the Management of Arterial Hypertension.


Each region represents the level of your blood pressure test result is in, compared to the 2007 ESH-ESC classifications, and can alert you and your healthcare professional when your blood pressure is higher than usual.

The higher the indication bars on the meter screen show, the greater the risk of the hypertension has. Please consult your healthcare professional if you receive high blood pressure continuously.


Blood pressure range represented by color of region on monitor.

REGION	WHAT IT MEANS
GREEN	Appears when your systolic pressure is below 130mmHg and your diastolic pressure is below 85mmHg.
YELLOW	Replace Appears when your systolic pressure is equal to or higher than 140mmHg, or your diastolic pressure is equal to or higher than 90mmHg.
ORANGE	Appears when your systolic pressure is equal to or higher than 160mmHg, or your diastolic pressure is equal to or higher than 100mmHg.
RED	Appears when your systolic pressure is equal to or higher than 180mmHg or your diastolic pressure is equal to or higher than 110mmHg.

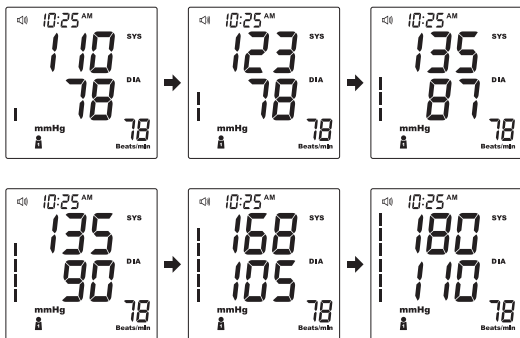For Example:

Error Messages

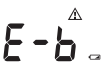MESSAGE	CAUSE	WHAT IT MEAN
	Big arm movement or vibration and unable to detect the pulse during the measurement.	The meter will have a second measurement automatically. Try not to move the arm and remain still during measurement.
	Inflation or pressure error.	Please contact local customer service for help.
	Blood pressure measurement error.	Refit cuff tightly and correctly. Relax and repeat the measurement. If error still remains, contact local customer service for help.
	Problems with the meter.	Review the instructions and repeat the test. If the meter still does not work, please contact the local customer service for help.
		
	Batteries are too low.	Repeat with new batteries or input AC adapter.

Troubleshooting

1. If nothing is displayed after pressing  :

POSSIBLE CAUSE	WHAT TO DO
Batteries exhausted.	Replace the batteries.
Batteries incorrectly installed or absent.	Check that the batteries are correctly installed.

2. If the heart rate is higher/lower than user's average:

POSSIBLE CAUSE	WHAT TO DO
Movement during measurement.	Repeat measurement.
Measurement taken just after exercise.	Rest at least 30 minutes before repeating measurement.

3. If the result is higher/lower than user's average measurement:

POSSIBLE CAUSE	WHAT TO DO
May not be in correct position while measuring.	Adjust to the correct position to measure.
Blood pressure naturally varies from time to time.	Keep in mind for next measurement.

4. If the cuff inflates again while measuring:

POSSIBLE CAUSE	WHAT TO DO
Cuff is not fastened.	Fasten the cuff again.
If user's blood pressure is higher than the pressure the device has inflated, the device will automatically increase the pressure and start to inflate again. Stay relaxed and wait for the measurement.	

DETAILED INFORMATION

Reference Values

Clinical studies show that adult diabetes is often accompanied by elevated blood pressure. People with diabetes can reduce their heart risk by managing their blood pressure along with diabetes treatment^{*1}. Monitoring your routine blood pressure trend helps you to know your body condition. Human blood pressure naturally increases after reaching middle age. This symptom is a result of continuous ageing of the blood vessels. Further causes include obesity, lack of exercise and cholesterol (LDL) adhering to the blood vessels. Rising blood pressure accelerates hardening of the arteries, and the body becomes more susceptible to apoplexy and coronary infarction.

The recommended blood pressure range is as below:

Classification	Systolic Pressure (mmHg)	Diastolic Pressure (mmHg)
Hypotension ^{*2}	< 90	< 60
Normal ^{*3}	< 120	< 80
Elevated ^{*3}	120 – 129	< 80
Stage 1 Hypertension ^{*3}	130 – 139	80 – 89
Stage 2 Hypertension ^{*3}	≥ 140	≥ 90
Hypertensive Crisis ^{*3}	> 180	> 120

*1: National Heart, Lung, and Blood Institute, Diseases and Conditions.

*2: The Seventh Report of the Joint National Committee on Prevention, Detection, Evaluation, and Treatment of High Blood Pressure. NIH Publication. 2003. No. 03-5233.

*3: New ACC (American College of Cardiology) and AHA (American Heart Association) guidelines for the detection, prevention, management and treatment of high blood pressure (2017).

SPECIFICATIONS

SYSTEM PERFORMANCE	
Power Source	Four 1.5V AA alkaline batteries
Size of Monitor w/o Cuff	150 (L) x 103 mm (W) x 74 mm (H), 320g without batteries
Memory	Maximum 120 memory records
Power Saving	Automatic power off if system is idle for 3 minutes
System Operating Condition	41 °F to 104 °F (5 °C to 40 °C), 15% to 93% RH
Monitor Storage Condition	-13 °F to 158 °F (-25 °C to 70 °C), below 95% RH
Power Supply Input	DC +6V / 1A (max) via Power Plug

BLOOD PRESSURE MEASUREMENT PERFORMANCE	
Pressure Range	0 - 300 mmHg
Measurement Unit	mmHg or KPa
Systolic Measurement Range	60 mmHg to 255 mmHg
Diastolic Measurement Range	30 mmHg to 195 mmHg
Pulse Rate Measurement Range	40 to 199 beats / minute
Accuracy of Pressure	±3 mmHg or ±2% of reading
Accuracy of Pulse Rate	±4% of reading

This device has been tested to meet the electrical and safety requirements of: IEC/EN 60601-1, IEC/EN 60601-1-2.

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Distributed by ForaCare, Inc./ Distribuido por ForaCare, Inc.