SootheNeb

NBL100 Mesh Nebulizer Cleaning Guide

This nebulizer offers a unique and advanced mesh technology that delivers your medication without the need of heating, and efficiently provides an exceptional treatment delivery system.

By following these simple seven steps, you will improve the longevity and functionality of your new SootheNeb Mesh Nebulizer. Prior to cleaning, make sure you have some **distilled water** on hand, which can be purchased at your local grocery store, and ensure that the device has completed the last medicine delivery cycle and is completely turned off.


Distributed by ForCare, Inc.
893 Patriot Dr., Suite D
Moorpark, CA 93021 USA
Products made in Taiwan
Toll Free: 1-888-307-8188 (8:30am-5:00pm PST, Monday-Friday)
For assistance outside of these hours, please contact your healthcare professional.

1 Remove the accessories and the mesh cap from the main unit but keep the medication cup attached.

2 Pour out any remaining medication.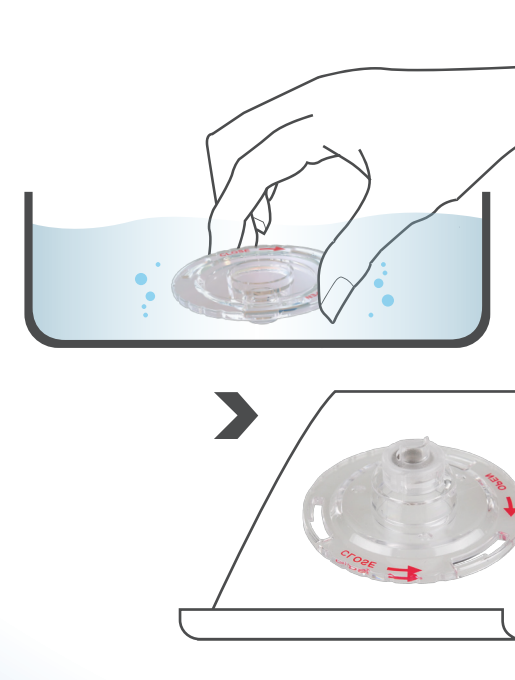


3 **Submerge Only** the **MESH CAP (not the medication cup)** in distilled water for about 30 seconds. Then, carefully **DRY** with a clean paper towel.

4 Add a small amount (approximately 1 ml) of distilled water into the medication cup.

5 Twist the mesh cap back on to the medication cup then press the on button to nebulize the distilled water for 1 to 2 minutes to remove residual medication.

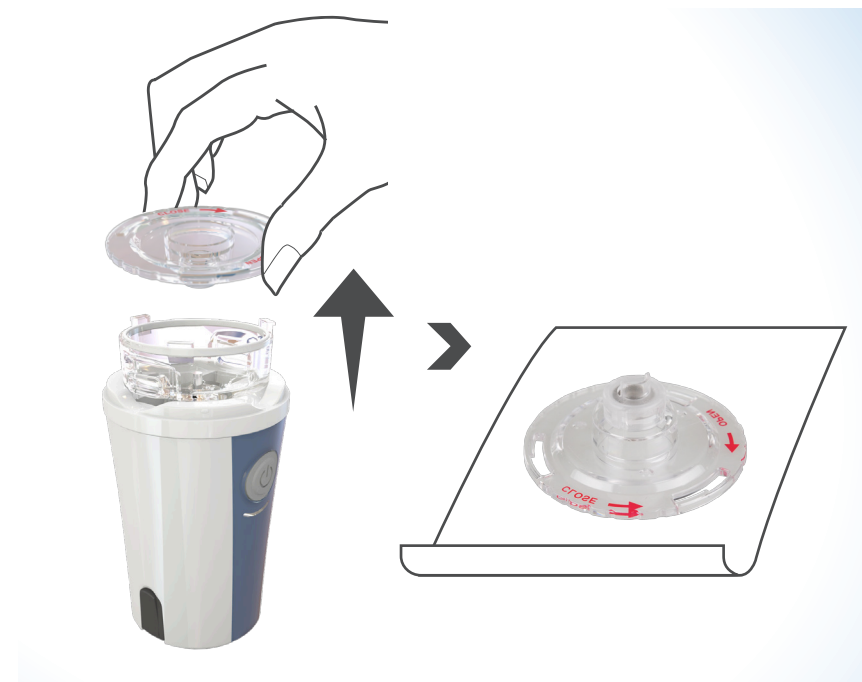6 Turn off the device after it's done then remove the medication cup to air **DRY** on a clean paper towel.

7 Once **all components** are **DRY**, assemble device and store in a clean environment.

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Distribuido por ForaCare, Inc.