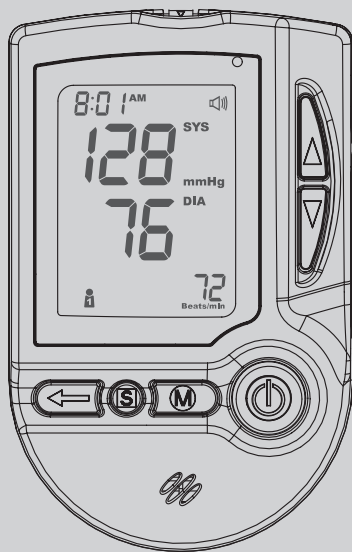
FORA[®] P20

BLOOD PRESSURE MONITORING SYSTEM

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Owner's Manual
Manual del Propietario

Dear **FORA[®] P20** System Owner:

Thank you for purchasing the **FORA P20** Blood Pressure Monitoring System. This manual provides important information which helps you to operate this system properly. Before using this product, please read the following contents thoroughly and carefully.

According to the clinical studies from American Diabetes Association, elevated blood pressure often accompanies adult diabetes patients. These studies also suggest that diabetes patients could reduce the risk of cardiovascular diseases by managing their blood pressure. With the use of this **FORA P20** Blood Pressure Monitoring System, you can easily monitor your blood pressure by yourself. In addition, this system can help you and your healthcare professionals to monitor and adjust your treatment plans, and keep your blood pressure under control.

If you have other questions regarding this product, please contact the place of purchase.

IMPORTANT SAFETY INSTRUCTIONS

READ BEFORE USE

1. Use this device **ONLY** for the intended use described in this manual.
2. Do **NOT** use accessories which are not specified by the manufacturer.
3. Do **NOT** use the device if it is not working properly or damaged.
4. Do **NOT** use under any circumstances on neonates or infants.
5. This device does **NOT** serve as a cure for any symptoms or diseases. The data measured are for reference only. Always consult your doctor to have the results interpreted.
6. Keep the equipment and its flexible cord away from hot surfaces.
7. Do **NOT** apply the cuff to areas other than the place directed.
8. Use of this instrument in a dry environment, especially if synthetic materials are present (synthetic clothing, carpets etc.) may cause damaging static discharges that may cause erroneous results.
9. Do not use this instrument in close proximity to sources of strong electromagnetic radiation, as these may interfere with the accurate operation.
10. Proper maintenance and periodic calibration are essential to the longevity of your device. If you are concerned about your accuracy of measurement, please contact local customer service for help.

KEEP THESE INSTRUCTIONS IN A SAFE PLACE

TABLE OF CONTENTS

BEFORE YOU BEGIN	06
Intended Use	06
Test Principle	06
Contents of System	07
Monitor Overview	08
Screen Display	09
Speaking Function	10
SETTING THE MONITOR	13
TESTING YOUR BLOOD PRESSURE	17
Before Measurement	17
Taking Measurements	20
MONITOR MEMORY	24
Reviewing Test Results	24
DOWNLOADING RESULTS	25
Data Transmission Via Cable (For FORA P20v only)	26
Data Transmission Via Bluetooth (For FORA P20b only)	27
MAINTENANCE	29
Battery	29
Using AC Adapter	31
Caring for Your Monitor	32
SYSTEM TROUBLESHOOTING	33
Error Messages	33
Troubleshooting	34
DETAILED INFORMATION	35
Reference Values	35
SPECIFICATIONS	36
FEDERAL COMMUNICATIONS COMMISSION (FCC) STATEMENT	37

BEFORE YOU BEGIN

Intended Use

The FORA P20 Blood Pressure Monitoring System is intended to be used to measure the systolic and diastolic blood pressure and pulse rate by using a non-invasive technique in which an inflatable cuff is wrapped on the upper arm. This system should only be used for the testing of people who are 18 years old or older.

This monitor has some speaking function but is not intended for use by the visually impaired.

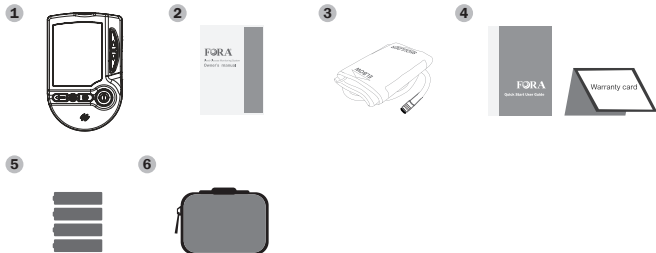
Test Principle

Blood pressure is measured non-invasively at the arm based on oscillometric method. Blood pressure measurements determined with this device are equivalent to those obtained by a trained observer using the cuff/stethoscope auscultation method, within the limits prescribed by the American National Standard, Electronic or automated sphygmomanometers.

This device is NOT able to take measurements in the presence of common arrhythmia, such as arterial or ventricular premature beats or arterial fibrillation. It may produce reading error.

Contents of System

Your new **FORA P20** system kit includes:


- 1 Monitor
- 2 Owner's Manual
- 3 Pressure Cuff (arm type)
- 4 Quick Start User Guide /
Warranty Card
- 5 Batteries
- 6 Protective Wallet

NOTE


If any items are missing from your kit or opened before use, please contact local customer service or place of purchase for assistance.

Monitor Overview

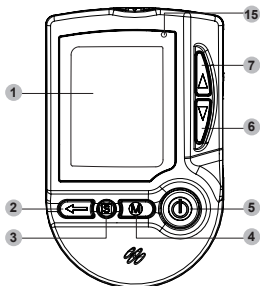
Front

Right Side

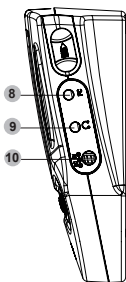Left Side

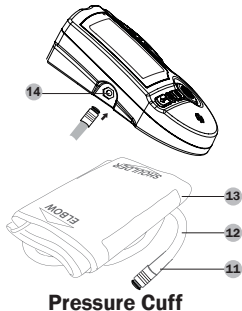1 Display Screen

2 Repeat Button

(for FORA P20v only)

Press to repeat the voice instructions.

Repeat/Pairing Button

(for FORA P20b only)

Press to repeat the voice instructions/pair with personal device.

3 S Button

4 M Button

5 On/Off Button

6 Down Button

7 Up Button

8 Data Port

(for FORA P20v only)

9 Earphone Jack

10 DC Adapter Port

11 Air Plug

12 Air Tube

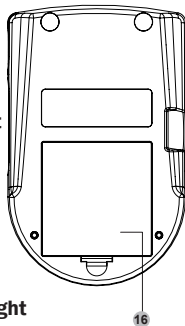
13 Pressure Cuff

14 Air Jack


15 BT Indication Light

16 Battery Compartment

Back

Screen Display

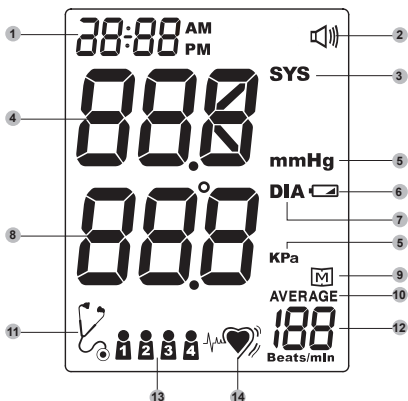- | | |
|-----------------------------|-----------------------|
| 1 Date and Time | 10 Three Time Average |
| 2 Volume Symbol | 11 Stethoscope Symbol |
| 3 Systolic Pressure Symbol | 12 Pulse Rate |
| 4 Systolic Pressure Value | 13 User Number |
| 5 Unit of Measurement | 14 Pulse Rate Symbol |
| 6 Battery Symbol | |
| 7 Diastolic Pressure Symbol | |
| 8 Diastolic Pressure Value | |
| 9 Memory Mode Symbol | |

Speaking Function

FORA P20 “speaks” aloud with step-by-step instructions to guide you through the process of using the meter. The following table tells you when and what the monitor “speaks”.

WHEN does the monitor say?	WHAT does the monitor say?
Setting up the monitor	
Setting the year	You are now in setting mode. The year is (number).
Setting the month	The month is (month).
Setting the date	The date is the (number)th.
Setting the hour format	The time is (12/24) hour mode.
Setting the hour	The hour is (number) PM/AM.
Setting the minutes	The minutes are (number).
Setting the blood pressure unit	Blood pressure unit is (millimeters of mercury / kilopascals).
Setting the voice volume	Voice volume is (number) / Voice is turned off.
Deleting the memory	<ol style="list-style-type: none">1. Delete memory record function. Press memory key to clear all records.2. Are you sure you want to clear all records? Press memory key again to clear all records.3. All memory records have been deleted.

Selecting language	Thank you for using FORA TeleHealth products.
At the end of monitor setting	Today's date is (day) (month) (date)th, (year). The time is (hour minute) AM/ PM. Blood pressure unit is (millimeters of mercury / kilopascals). Voice volume is (number). Have a nice day.
Using the monitor	
When turning on the monitor	Thank you for using FORA TeleHealth products. The user's number is (number). Please select user's number.
After selecting the user number.	The user's number is (number). Please relax during measurement.
When the test is completed. (result appears on display)	Your systolic blood pressure is (number) (millimeters of mercury/ kilopascals). Your diastolic blood pressure is (number) (millimeters of mercury/ kilopascals). Your heart rate is (number) beats per minute.

<p>Recall the Stored Test Results</p>	<ol style="list-style-type: none"> 1. Thank you for using FORA Tele-Health products. Today's date is (day) (month) (date)th, (year). The time is (hour minute) AM/PM. You are now in memory mode. The user's number is (number). Please select user's number. 2. The user's number is (number). 3. Memory record for (month) (date) th, (year), (hour minute) AM/PM. Your systolic blood pressure was (number) millimeters of mercury/kilopascals. Your diastolic blood pressure was (number) millimeters of mercury/kilopascals. Your heart rate was (number) beats per minute.
<p>If no results stored in memory</p>	<p>No memory record available.</p>
<p>When the monitor is turned off</p>	<p>Have a nice day.</p>
<p>When entering the transmission mode.</p>	<p>Connecting to your computer.</p>

NOTE

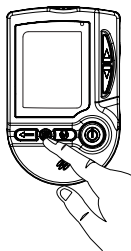
If you prefer to listen to the voice messages by earphone, plug a pair of earphones into the earphone jack on the side of your meter.

SETTING THE MONITOR

Before using your monitor for the first time, check and update these settings. Make sure you complete the steps below and save the desired settings.

► Entering the setting mode


Start with the monitor off. Press **S** until the monitor turns on.

Step 1

Setting the date.

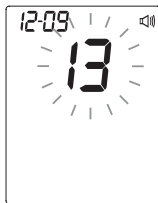
With the year flashing, press **▲▼** to select the correct year. Press **S** to set.

With the month flashing, press **▲▼** to select the correct month. Press **S** to set.

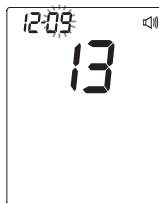
With the day flashing, press ▲▼ to select the correct day. Press S to set.

Step 2

Set the time format.

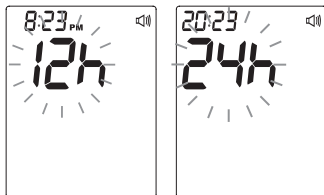
Press ▲▼ to select the desired time format -- 12h or 24h. Press S to set.


Step 3

Set the time

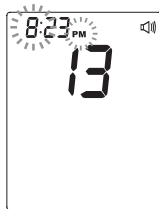With the hour flashing, press ▲▼ to select the correct hour. Press S to set.

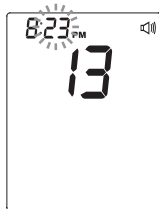
With the minute flashing, press ▲▼ to select the correct minute appears. Press S to set.

Step 4

Set the unit of measurement.

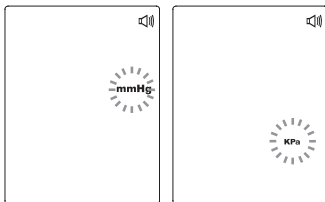Press **▲▼** to select **mmHg** or **Kpa**. Press **S** to set.


Step 5

Setting the speaking volume.

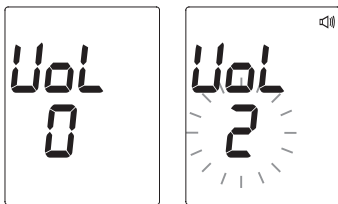There are seven (7) speaking volume options you can choose from.

Press **▲▼** to select the desired speaking volume. Press **S** to set.

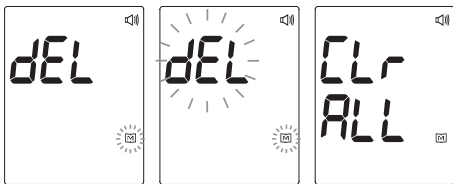
Volume 0 indicates that the speaking function is turned off, and will not display “”. Volume 1 to 7 indicates speaking volume from low to high, and “” will be displayed during testing.

Step 6

Delete the memory.

While “dEL” and a blinking “M” symbol appear on the display, if you do not wish to delete the saved results, press **S** to skip this step.

If you wish to delete ALL the results, press **M** twice. “CLr” and “M” are displayed on the monitor, which indicates that all results have been deleted.


Step 7

Choosing a language.

Press **▲▼** to select L1/L2. The default language for the monitor is L1, which is English. Press **S**.

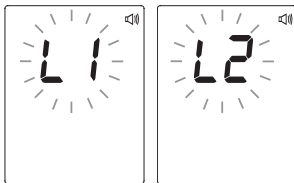Then the monitor will turn off automatically or you can press **⏻** to turn off the monitor. The monitor repeats all your settings before shutdown.

Congratulations! You have completed all settings!

NOTE

- These parameters can **ONLY be changed** in the setting mode.
- If the monitor is idle for 3 minutes during the setting mode, it will turn off automatically.

TESTING YOUR BLOOD PRESSURE

This monitor provides you three different ways to measure your blood pressure. Select from the operations below:

- **Single measurement**
Perform an individual blood pressure measurement.
- **Averaging measurement**
Automatically perform three (3) consecutive blood pressure measurements and display the average result at the end.
- **Auscultatory measurement**
Only trained health-care professionals may use this mode to measure blood pressure manually with a stethoscope by listening to the Korotkoff sound.

Before Measurement

- Avoid caffeine, tea, alcohol and tobacco for at least 30 minutes before measurement.
- Wait for 30 minutes after exercising or bathing before measurement.
- Sit or lie down for at least 10 minutes before measurement.
- Do not measure when feeling anxious or tense.
- Take a 5-10 minute break between measurements. This break can be longer if necessary, depending on your physical conditions.
- Keep the records for your doctor as reference.
- Blood pressure varies between each arm. Always measure your blood pressure on the same arm.

► Fitting the Cuff Correctly

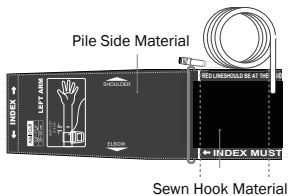
Step 1

Connect the air plug of the tubing to the air jack of the monitor.


Step 2

Assemble the cuff as shown on the right. The smooth surface should be inside the cuff loop and the metal D-ring should not touch your skin.

Step 3

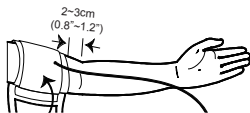Stretch your left (right) arm in front of you with your palm facing up. Slide and place the cuff onto your arm with the air tube and artery mark region (in red) toward the lower arm.


Wrap and tighten the cuff above your elbow. The red line on the edge of the cuff should be approximately 0.8 to 1.2 inches (2 to 3 cm) above your elbow. Align the tube over the main arteries on the inside.

Step 4

Leave a little free space between the arm and the cuff, you should be able to fit 2 fingers between them. Clothing must not restrict the arm. Remove all clothing covering or constricting the measurement arm.

Step 5

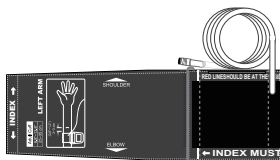Press the hook material firmly against the pile material. The top and bottom edges of the cuff should be tightened evenly around your upper arm.

► The range index of cuff should fall into this range.

► Proper Measurement Position

Step 1

Sit down for at least 10 minutes before measuring.

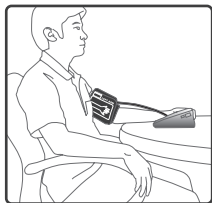
Step 2

Place your elbow on a flat surface. Relax your hand with the palm facing up.

Step 3

Make sure the cuff is about the same height as the location of your heart.

Press . Remain still and do not talk or move during the measurement.

WARNING

If the cuff is relatively lower/higher than the heart, the obtained blood pressure value could be higher/lower than the actual value. A 15 cm difference in height may result in an error around 10 mmHg.

Step 4

Measurement is in progress.

After the monitor is turned on, the cuff will begin to inflate automatically.

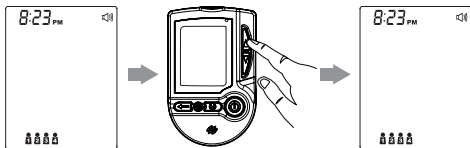
Taking Measurements

► Taking a Single Measurement

Always apply the pressure cuff before turning on the monitor.

Step 1

Press . All the LCD symbols will appear. Press   to select user “   ” (numbers from 1 to 4) whose test reading will be stored in memory. Then the cuff will begin to inflate automatically.

Step 2

The heart symbol “” will flash when a pulse is detected during the inflation.

Step 3

After the measurement, the monitor displays the systolic pressure, diastolic pressure and pulse rate.

Step 4

Press  to switch off. Or it will switch off automatically after left idle for 3 minutes.

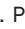
NOTE

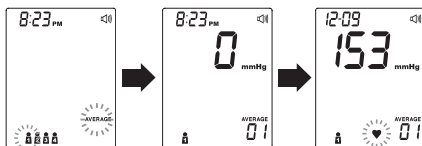
- If you press  during measurement, the monitor will be turned off.
- If the pulse rate symbol is shown as “” instead of “”, this indicates that the monitor has detected an irregular heart beat.

► Averaging Measurement Mode

Always apply the pressure cuff before turning on the monitor.

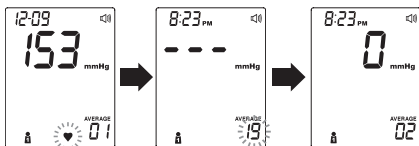
Step 1

Press and hold  for three seconds. The monitor will turn on and enter the averaging mode. Press   to select user “1 2 3 4” (numbers from 1 to 4) whose test reading will be stored in memory and press  to confirm. Then the cuff will begin to inflate automatically.

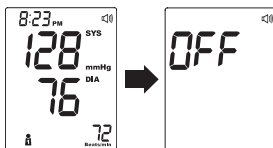
Step 2

After the first measurement is completed, the monitor will start counting down before the second measurement begins. The number on the right represents the remaining countdown between each measurement. The monitor will take three (3) measurements consecutively with intervals of 20 seconds.

Step 3

After taking three measurements, the results are averaged to produce the blood pressure measurement. Press  to turn off the monitor.

► Auscultatory Mode

Only trained healthcare professionals may perform this manual blood pressure measurement.

This manual method involves applying a stethoscope to patient's arm and listening to the pulse while the air is slowly let out again.

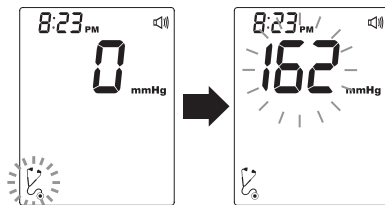
The systolic pressure is the maximum pressure in an artery at the moment when the heart is beating and pumping blood through the body. The diastolic pressure is the lowest pressure in an artery in the moments between beats when the heart is resting.

Step 1

Place a stethoscope on the patient's arm where there is a pulse. Apply the cuff to the patient's upper arm and hold in place with Velcro.

Step 2

Press **⏻** and wait for the user number to appear. Press **M** to switch to the auscultatory mode. A stethoscope symbol will appear on the display with a beep. The cuff begins to inflate automatically.

Step 3

After reaching the cuff pressure, the deflation begins. The operator can adjust the inflation pressure if necessary. During the measurement, press and hold  button at any time to keep inflating, release at the desired pressure.

Step 4

The systolic pressure is measured when the operator first hears the pulse. Take the reading according to the value on the display. This reading indicates the patient's systolic pressure.

Step 5

The diastolic pressure is measured from the moment the operator is unable to hear the sound of the pulse. Take the reading according to the value on the display. This reading indicates the patient's diastolic pressure.

MONITOR MEMORY

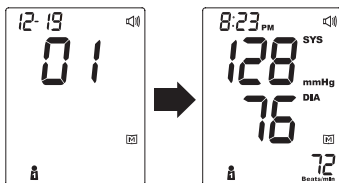
Your monitor stores the 400 most recent test results along with respective dates and times in its memory. To recall the memory, **start with the monitor off**.

Reviewing Test Results

Step 1

Press and release **M**.

“**M**” will appear on the display. The user symbol (numbers from 1 to 4) appears first, press **▲▼** to select which one you used to store in the memory.

Step 2

Press **▲▼**.

Review all test results stored in the monitor.

Step 3

Exit the memory.

Press **⏻** and the monitor will turn off.

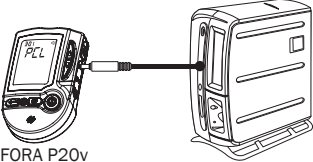
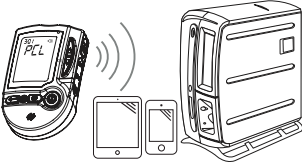
NOTE

- Any time you wish to exit the memory, press **⏻** or leave it idle for 3 minutes. The monitor will turn off automatically.
- If using the monitor for the first time, “—” displays when you recall the test results or review the average result. This indicates that there is no test result in the memory.

DOWNLOADING RESULTS

There are two subtypes of **FORA P20** system available. Each one of them offers a different way to transmit, allowing users to choose their own desired way to manage their test results. These feature functions are listed below:

Model	Data Transmission Function
 <p>FORA P20v</p>	Test results can be viewed on the PC through the cable connection.
 <p>FORA P20b</p>	Test results can be viewed on PC, smart phone or tablet through a wireless connection by using Bluetooth technology.

For FORA P20v, the Health Care Software System and an interface cable are required before installation. To learn more about the Health Care Software System or to obtain an interface cable separately, please contact your place of purchase.

Data Transmission Via Cable (For FORA P20v only)

Step 1

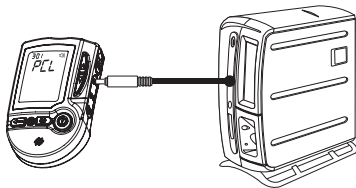
Install Software.

To download the Health Care Software System, please visit the FORA Care, Inc. website at <http://www.foracare.com/support.html> and select the “Software & App” to download the “Health Care Software System” software. Follow the online instructions to install the software to your computer.

Step 2

Connect the RS232 interface cable to a serial port on PC.

With the monitor turned off, connect the RS232 interface cable to the data port located at the side of the monitor. “PCL” will appear on the display.

Step 3

Data transmission.

Follow the instructions provided in the software to transmit data (results and their corresponding dates and times). Remove the cable or press

 and the monitor will turn off.

NOTE

The monitor is unable to perform a test while connected to the PC.

Data Transmission Via Bluetooth (For FORA P20b only)

You can transmit your data from the meter to your devices (e.g. smart phone, tablet, PC...) via Bluetooth. Please contact your local customer service or place of purchase for assistance. Please note that you must pair your meter to your device before transmitting data.

► Pairing with your mobile device

Step 1

Press and hold  on the meter until the Bluetooth indicator starts to flash.

Step 2

On your mobile device, turn on the Bluetooth function and follow the instruction on your APP to pair the device (e.g., scan for the meter to add it into APP).

Step 3

After successfully pairing the APP to your meter, the Bluetooth function on the meter will then start to transmit your data to your APP.

► Bluetooth indicator on the meter

BLUETOOTH INDICATOR	STATUS
Flashing Blue	The Bluetooth function is enabled and waiting for connection.
Solid Blue	The Bluetooth connection is established.

NOTE

- While the meter is in transmission mode, it will be unable to perform a test.
- The meter is compatible with the following devices, iOS (6 or above) and Android system (4.3 or above). Make sure device is in proper receiving range of the meter before transmitting data.
- A compatibility issue between your mobile device and the meter might occur due to the way Bluetooth functionality is implemented by the various mobile device manufacturers.

MAINTENANCE

Battery

Your monitor comes with four (4) 1.5V AA size alkaline batteries.

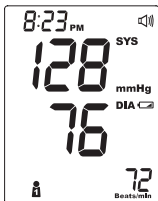
► Low Battery Signal

The monitor will display either of the two messages below to alert you when the monitor power is getting low.

Step 1

The “” symbol appears along with display messages:

The monitor is functional and the result remains accurate, but it is time to change the batteries.

Step 2

The “” symbol appears with E-b and low:

There is not enough power to do a test. You must change the batteries immediately.

► Replacing the Battery

To replace the batteries, make sure that the monitor is turned off.

Step 1

Press the edge of the battery cover and lift it up to remove.

Step 2

Remove the old batteries and replace with four 1.5V AA size alkaline batteries.

Step 3

Close the battery cover.

NOTE

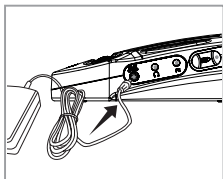
- Replacing the batteries does not affect the test results stored in memory.
- As with all small batteries, these batteries should be kept away from small children. If swallowed, promptly seek medical assistance.
- Batteries might leak chemicals if unused for a long time. Remove the batteries if you are not going to use the device for an extended period (i.e., 3 months or more).
- Properly dispose of the batteries according to your local environmental regulations.

Using AC Adapter

► Connect AC adapter to the monitor


Step 1

Connect AC adapter plug to AC adapter jack of the monitor.

Step 2

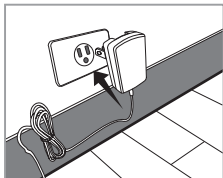
Plug AC adapter power plug into an electrical outlet. The monitor is ready for use.

► Remove AC adapter from the monitor


Step 1

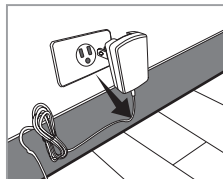When the monitor is off, remove AC adapter power plug from the electrical outlet.

Step 2

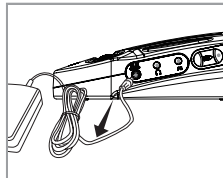Disconnect AC adapter plug from AC adapter jack on the monitor.

Caring for Your Monitor

To prevent the meter from dirt, dust or other contaminants, please wash and dry your hands thoroughly before use.

► **Cleaning**

- To clean the monitor exterior, wipe it with a cloth moistened with tap water or a mild cleaning agent, then dry the device with a soft dry cloth. Do **NOT** flush it with water.
- Do **NOT** use organic solvents to clean the monitor.
- Do **NOT** wash the pressure cuff.
- Do **NOT** iron the pressure cuff.

► **Monitor Storage**

- Storage condition: -13°F to 158°F (-25°C to 70°C), below 95% relative humidity.
- Always store or transport the monitor in its original storage case.
- Avoid dropping and heavy impact.
- Avoid direct sunlight and high humidity.

► **Monitor Disposal**

The batteries in this used meter should be removed and the meter should be disposed in accordance with local regulations.

SYSTEM TROUBLESHOOTING

If you follow the recommended actions but the problem persists, or error messages other than the ones below appear, please contact local customer service. Do not attempt to repair by yourself and never try to disassemble the monitor under any circumstances.

Error Messages

MESSAGE	CAUSE	WHAT TO DO
E-1	Inflation or pressure error.	Please contact local customer service for help.
E-2	Deflation error.	
E-4	Blood Pressure measurement error.	Refit cuff tightly and correctly. Relax and repeat measurement. If error still exists, please contact the local customer service for help.
E-b	Battery is too low.	Replace the batteries or connect 6V DC adaptor.
E-E, E-A, E-3, E-C	Problem with the monitor.	Review the instructions and repeat the test. If the monitor still does not work, please contact the local customer service for help.

Troubleshooting

1. If nothing is displayed after pressing  :

POSSIBLE CAUSE	WHAT TO DO
Batteries exhausted.	Replace the batteries.
Batteries incorrectly installed or absent.	Check that the batteries are correctly installed.

2. If the heart rate is higher/lower than user's average:

POSSIBLE CAUSE	WHAT TO DO
Movement during measurement.	Repeat measurement.
Measurement taken just after exercise.	Rest at least 30 minutes before repeating measurement.

3. If the result is higher/lower than user's average measurement:

POSSIBLE CAUSE	WHAT TO DO
May not be in correct position while measuring.	Adjust to the correct position to measure.
Blood pressure naturally varies from time to time.	Keep in mind for next measurement.

4. If the cuff inflates again while measuring:

POSSIBLE CAUSE	WHAT TO DO
Cuff is not fastened.	Fasten the cuff again.
If user's blood pressure is higher than the pressure the device has inflated, the device will automatically increase the pressure and start to inflate again. Stay relaxed and wait for the measurement.	

DETAILED INFORMATION

Reference Values

Clinical studies show that adult diabetes is often accompanied by elevated blood pressure. People with diabetes can reduce their heart risk by managing their blood pressure along with diabetes treatment*¹.

Knowing your routine blood pressure trend tells you whether your body is in good condition or not. Human blood pressure naturally increases after reaching middle age. This symptom is a result of continuous aging of the blood vessels. Further causes include obesity, lack of exercise and cholesterol (LDL) adhering to the blood vessels. Rising blood pressure accelerates hardening of the arteries, and the body becomes more susceptible to apoplexy and coronary infarction. The recommended blood pressure range is presented below:

Classification	Systolic Pressure (mmHg)	Diastolic Pressure (mmHg)
Hypotension* ²	< 90	< 60
Normal* ³	< 120	< 80
Pre-hypertension* ³	120–139	80–89
Stage 1 Hypertension* ³	140–159	90–99
Stage 2 Hypertension* ³	≥ 160	≥ 100

References:

*1: National Heart, Lung, and Blood Institute, Diseases and Conditions

*2: The Seventh Report of the Joint National Committee on Prevention, Detection, Evaluation, and Treatment of High Blood Pressure. NIH Publication. 2003. No. 03-5233

*3: American Diabetes Association: The Diabetes-Heart Disease Link Surveying Attitudes, Knowledge and Risk (2002)

The values in this table are for reference only. They may be used as a guide in understanding non-invasive blood pressure measurements.

SPECIFICATIONS

SYSTEM PERFORMANCE	
Power Source	Four 1.5V AA alkaline batteries
Size of Monitor w/o Cuff	152 (L) x 99 mm (W) x 55 mm (H), 340g with batteries
Cuff Size	M (medium): 24-35 cm (9.4-13.8 inches) with air tube 80 cm
Memory	4 users with 100 records for each; total 400 Sets.
External Output	RS232-4Poles (For P20v), Bluetooth (For P20b)
Power Saving	Automatic power off if system is idle for 3 minutes (normal mode) or 5 minutes (RF mode)
System Operating Condition	41 °F to 104 °F (5 °C to 40 °C), below 93% RH
Monitor Storage Condition	-13 °F to 158 °F (-25 °C to 70 °C), 10% to 95% RH
Power Supply Input	DC +6V / 1A (max) via Power Plug
Expected Service Life	3 years

BLOOD PRESSURE MEASUREMENT PERFORMANCE	
Pressure Range	0 - 300 mmHg
Measurement Unit	mmHg or KPa
Systolic Measurement Range	50 mmHg to 250 mmHg
Diastolic Measurement Range	30 mmHg to 180 mmHg
Pulse Rate Measurement Range	40 to 199 beats / minute
Maximum Inflation Pressure	300 mmHg
Accuracy of Pressure	± 3 mmHg or $\pm 2\%$ of reading
Accuracy of Pulse Rate	$\pm 4\%$ of reading

This device has been tested to meet the electrical and safety requirements of: IEC/EN 60601-1, IEC/EN 60601-1-2.

Reference to Standards:

- EN 1060-1 /-3, NIBP-requirements
- IEC60601-1 General requirement for safety
- IEC60601-1-2 Requirements for EMC
- EN1060-4, NIBP clinical investigation
- AAMI / ANSI / IEC 80601-2-30, ANSI/AAMI/ISO 81060-2, NIBP requirements
- FCC part 15C

FEDERAL COMMUNICATIONS COMMISSION (FCC) STATEMENT

15.21

You are cautioned that changes or modifications not expressly approved by the part responsible for compliance could void the user's authority to operate the equipment.

15.105(b)

Federal Communications Commission (FCC) Statement

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

- 1) This device may not cause harmful interference and
- 2) This device must accept any interference received, including interference that may cause undesired operation of the device.

FCC RF Radiation Exposure Statement:

1. This Transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.
2. This equipment complies with FCC RF radiation exposure limits set forth for an uncontrolled environment. This equipment should be installed and operated with a minimum distance of 20 centimeters between the radiator and your body.

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Foracare, Inc.**