

FORA[®] iFORA Diabetes Manager Application

Getting Started

- 1 Download and install the latest app version of ' iFORA Diabetes Manager' on your phone.
- 2 Make sure that Bluetooth is enabled in smartphone settings.




- 3 Read the terms and conditions, then check the box next to "Agree" and press "OK"




Connecting a Device

- 1 Press and hold Bluetooth switch on the right side of the meter (5 seconds) until Bluetooth indicator starts to flash blue.
- 2 From your phone app, select 'Add' for TN'G Voice meter.

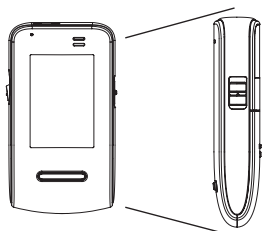


Bluetooth switch

- 3 This device will be added to 'Current Meters' list.

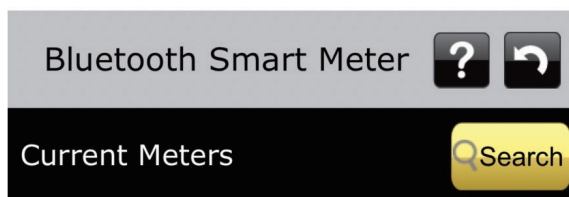


TNG VOICE

Delete

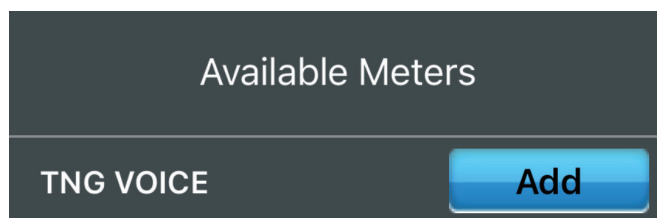



- 4 Hit the back arrow to exit the Bluetooth Smart Meter page. When prompted to save, press 'Yes'. data will be automatically imported.

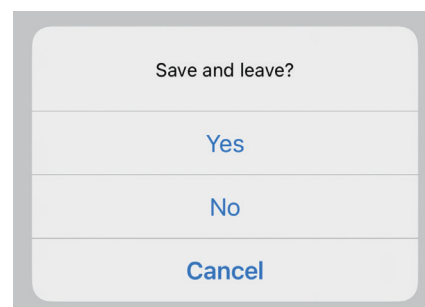

FORA® iFORA Diabetes Manager Application

Uploading Data to Telehealth Service

- 1 Launch iFORA Diabetes Manager. Go to the settings icon and find the “TeleHealth Service” page.
- 2 Select “ForaCare US” from the list under “Choose Your Telehealth Service”.

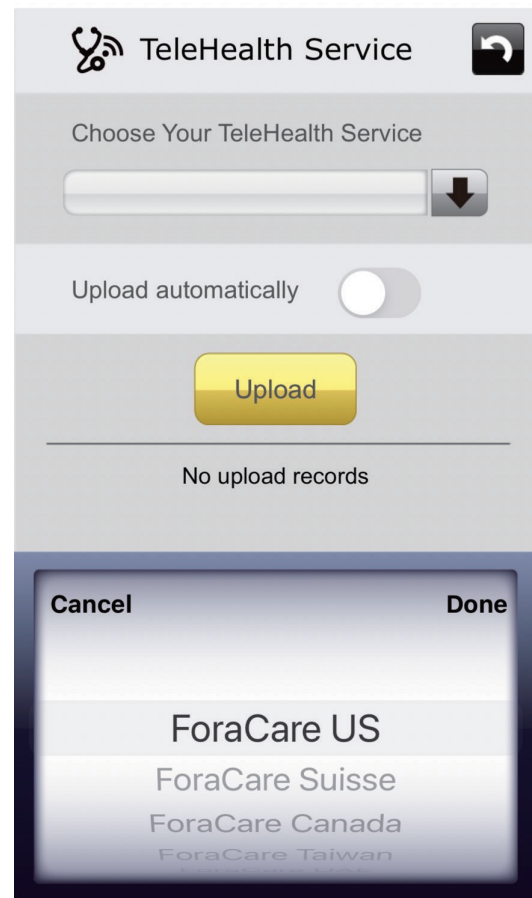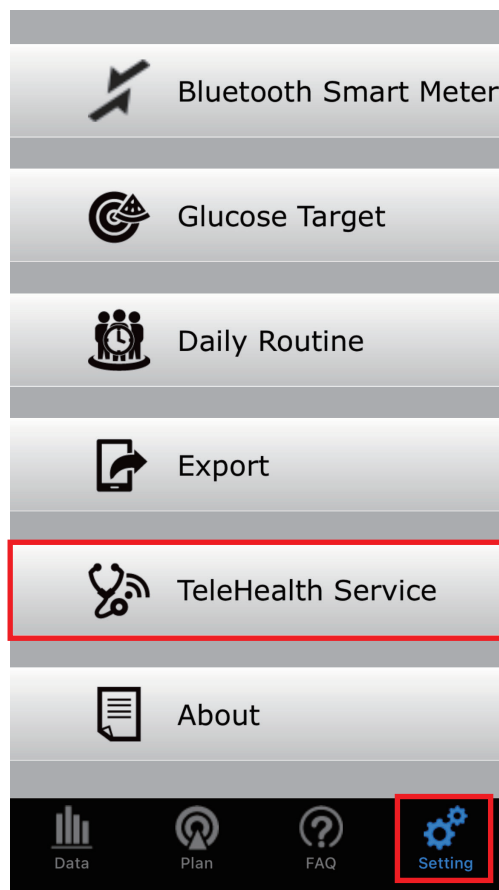

- 3 Switch on “Upload automatically” to automatically send data to the telehealth service.
- 4 Hit the back button to exit, and when prompted to save, press “Yes”. Data will be automatically uploaded each time your FORA device syncs with the app.

