



- 1 Wash your hands.

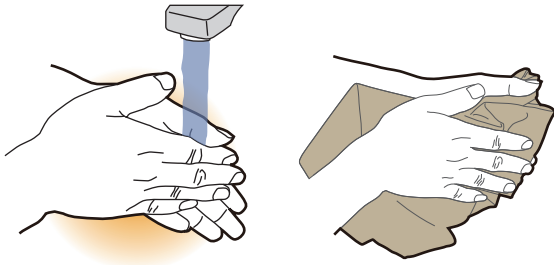


- 2 Insert lancet into FORA lancing device and close. Please refer to the *Lancing Device Set-up*

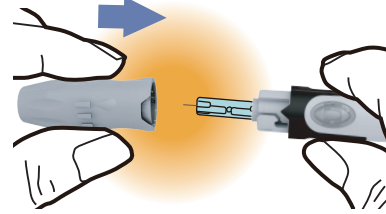


- 3 Insert test strip into the meter. The meter will turn on.



- 4 Press lancing device against side of finger and click orange release button.

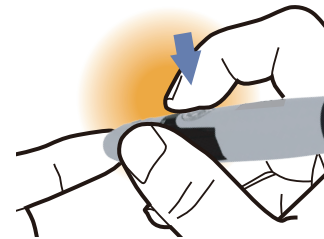

- 5 Apply the blood sample to the tip of the test strip. The test strip will absorb the sample of blood.

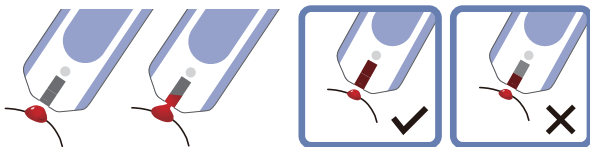

- 6 Wait 5 seconds for results.



- 7 Eject the test strip from the meter by sliding up the button on the left side of the meter.



Bluetooth Indicator

Flashing Blue:


The Bluetooth function is enabled and waiting for connection.

Solid Blue:


The Bluetooth connection is established.



- 1 Lavese sus manos.

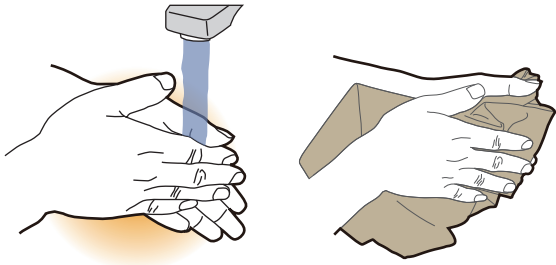


- 2 Inserte la lanceta en el dispositivo de punción FORA y ciérrelo. Consulte la configuración del dispositivo de punción para más información.

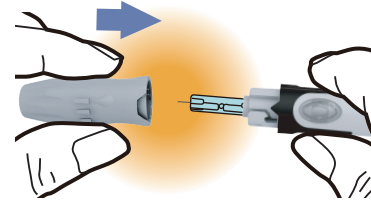


- 3 Inserte la tira reactiva para prender el medidor.



- 4 Presione el dispositivo de punción contra el costado del dedo y haga clic en el botón de liberación anaranjado.

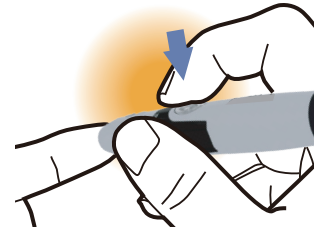

- 5 Aplique la muestra de sangre en la punta de la tira reactiva. La tira absorberá la muestra de sangre.

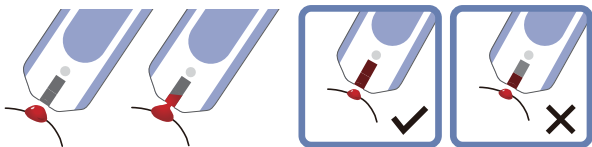

- 6 Espere 5 segundos para obtener resultados.



- 7 Expulse la tira reactiva del medidor deslizando hacia arriba el botón en el lado izquierdo del medidor.



Indicador de Bluetooth

Luz Azul Parpadeando:

La función Bluetooth está activada y esperando la conexión.

Luz Azul Solida:

La conexión Bluetooth está establecida.

