

# FORA<sup>®</sup> TN'G Voice-Settings


## Entering the Setting Mode

Start with the meter off (no test strip inserted). Open the battery compartment on the back and press the SET button.




### 1 Setting the date

With the year flashing, press Main button until the correct year appears. Press SET. Repeat with the month/day.

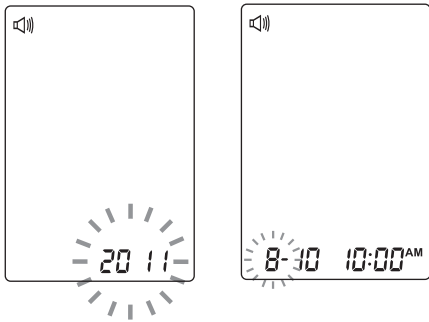


### 2 Setting the time format

Press Main button to select the desired time format --- 12h or 24h. Press SET.

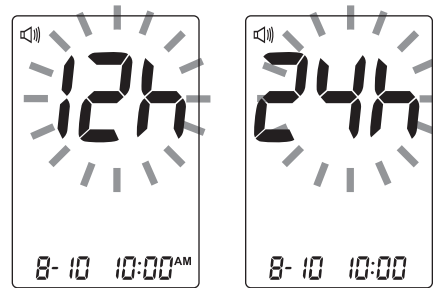


### 3 Setting the date

With the hour flashing, press MAIN button until the correct hour appears. Press SET. With the minute flashing, press Main button until the correct minute appears. Press SET.

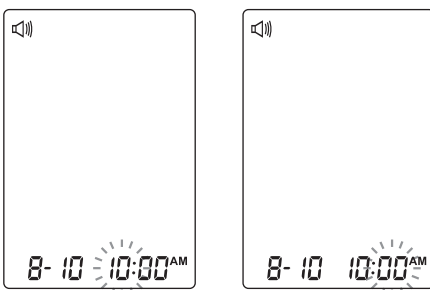

### 4 Delete Memory


Option to delete memory. Press Main button and select "no" to keep the results in memory then press SET to skip.

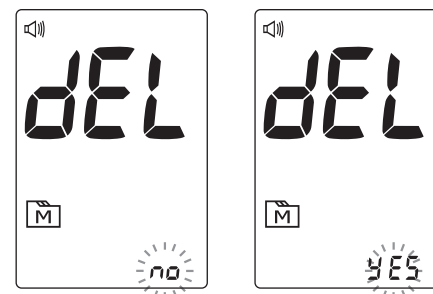

# FORA<sup>®</sup> TN'G Voice-Settings


## 5 Volume

There are seven (7) speaking volume options from 0 (off) to 7 (loudest). Press Main button until wanted speaking volume appears. To confirm your selection, press SET.

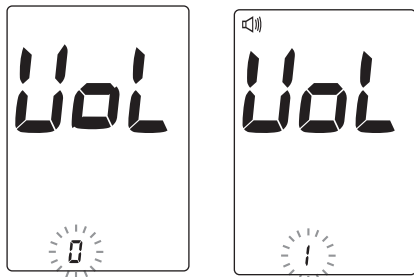


## 6 Language

Press Main button to select L1 (English), L2 (Spanish). Press SET.

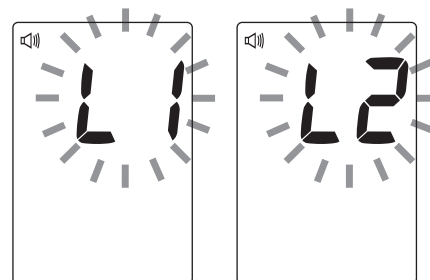


## 7 Alarms

Select the alarm number and then press Main button to change displayed message from "ON" to "OFF". Press SET to confirm and go to the next alarm setting.

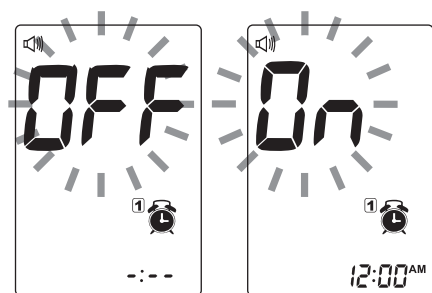

## 8 Setting the Auto-Send

Press Main button to select the auto send On. Press set.

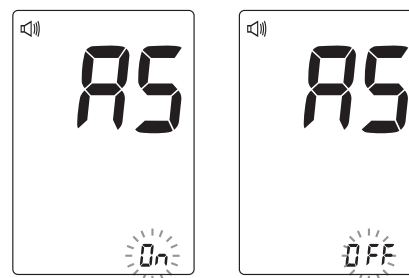

## General Information

- Meter will switch off in 2 minutes if not used.
- Meter transmits data automatically. To pair the device to the iFORA app, please refer to the iFORA Diabetes Manager Application guide.
- If you have questions about the accuracy of results, please contact customer service.